Evolsin® Narbenpflaster

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Evolsin® Scar sheet

The EVOLSIN® SCAR SHEET has been specially developed for old and new scars to reduce the visibility of higher performance scar blemishes and increase skin moisturizing. It is a safe and effective way and can be used in case of hypertrophic scars or raised and colored keloid scars.

The EVOLSIN® SCAR SHEET is designed to build a semi-occlusive barrier which improves hydration of the scar tissue. It also increases temperature in the scar tissue. This helps to activate the skin's own regeneration process and supports the remodelling of the scar. The scars become flatter, lighter and softer.

Clinical studies have shown **positive results after 8 weeks of consecutive use of at least 12 h a day**. First results are noticeable after 3 to 4 weeks. Using the patch for longer than 8 weeks may further improve the appearance of the scar. **Results may vary from person to person and from scar to scar.**

PRODUCT INFORMATION

- For: Surgical, Caesarean, Hypertrophic, Acne and Burn Scars
- Protective
- Reduce the discomfort of the scar to the touch
- Repositionable
- Water resistant
- Cut to size
- Dermatologically tested
- Clinically tested*

*Clinical and instrumental studies demonstrate a reduction in the visibility of scars and an increase in skin hydration

APPLICATION

In the case of a surgical intervention treatment can be started as soon as the wound has completely closed and healed and after sutures have been removed.

1. clean and dry the affected skin / scar area
2. cut the plaster to an appropriate size
3. stick the patch on and wait a few minutes until it fits perfectly
4. wear the patch for at least 12h per day. A patch can be used for several days until it no longer adheres to the skin.

WARNINGS

- Keep out of the reach of children.
- If swallowed, seek medical advice immediately and show the product box and/or leaflet.
- In the event of serious incidents, report to the manufacturer and the responsible authorities

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EUROSIREL S.p.A.