## Evolstin® Hämorrhoiden Gel

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CE 0481  
PZN 17150459

## Evolstin® Hemorrhoid Gel

- Treatment of external hemorrhoids.
- Relief of hemorrhoid symptoms such as burning, itching and pain caused by hemorrhoids.
- Relief of local irritation, fissures and external discomfort.

### GENERAL ADVICE:

Carefully read the instructions for use prior to using EVOLSTIN® HEMORRHOID GEL, even if you have used this medical device before. The information may have been amended to take account of new research and findings in the time since you opened your last package. This medical device can be obtained without a prescription. In spite of this, take care when using EVOLSTIN® HEMORRHOID GEL in order to obtain a good result. Keep these instructions for use in case you need to consult them again.

### APPLICATION:

EVOLSTIN® HEMORRHOID GEL is a medical device for the treatment of external hemorrhoids and other minor anorectal conditions such as anal itching and fissures. This protective gel reduces irritation, swelling, itching, pain and burning, and prevents contact of the irritated skin with aggravating liquid or stool from the rectum.

### DOSAGE AND USE (for adults):

Squeeze the tube of EVOLSTIN® HEMORRHOID GEL and apply a small amount on your finger. Apply the gel around the outside of your anus with your finger. Wash your hands thoroughly with warm water and soap after each application. Apply the gel to the affected area two to four times a day, preferably after a bowel movement, for 7-14 days.

### HOW IT WORKS:

EVOLSTIN® HEMORRHOID GEL forms a protective layer with excellent moisturizing and anti-inflammatory properties. The layer protects the area around the anus and restores the structural integrity of veins and normal blood flow to the area around the anus. It reduces inflammation and the risk of infection, not because it kills bacteria but because it prevents them from penetrating the skin. It helps to resolve a prolapsed Hemorrhoid before it grows large and produces a smoothing effect for the passage of hard stools.

### POSSIBLE SIDE EFFECT(S):

Irritation could be a side effect.

### USE DURING PREGNANCY AND/OR BREASTFEEDING:

No data is available about using EVOLSTIN® HEMORRHOID GEL during pregnancy or the breastfeeding period. Due to a lack of information EVOLSTIN® HEMORRHOID GEL is not recommended during the pregnancy or breastfeeding period.

### WARNINGS:

- Keep out of reach of young children.
- If you are suffering from hypersensitivity reactions, stop using EVOLSTIN® HEMORRHOID GEL.
- Do not use after the expiry date stated on the packaging.

### COMPOSITION:

Aqua, Glycerin, Betaine, Panthenol, Ammonium Acryloyldimethyltaurate/VP Copolymer, Laureth-9, Phenoxyethanol, Xanthan Gum, Allantoin, Ethylhexylglycerin, Sodium Carboxymethyl Betaglukan, Opuntia Ficus-Indica Stem Extract, Sodium Hyaluronate, Citric Acid, Alcohol.

### CONTENT: 40 ml

### STORAGE METHOD:

Store at room temperature (5-25°C) in a dry and dark place in the original container and out of reach of young children.

### FINAL DATE OF USE:

We advise against using EVOLSTIN® HEMORRHOID GEL after the expiry date, which is stated on the packaging after 'expiry date'. EVOLSTIN® HEMORRHOID GEL can be kept for 1 month after opening.

### MEDICAL DEVICE:

EVOLSTIN® HEMORRHOID GEL is a class IIA medical device in accordance with the Medical Device Directive 2007/47/EC. Date on which these instructions for use were last revised: 10-2020 V1.0

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