DynamicFX Rückenorthese

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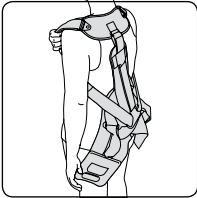
Functionality and effect:


- Counteraction of the kyphosis in the thoracic spine area by activating the torso muscles.
- Improves posture and brings pain relief by dynamically straightening the lumbar spine and thoracic spine.
- Relief and clearing of lordosis of the lumbar spine.
- Improvement of the body posture and strengthening of the muscles.
- Stabilisation of the achieved straightening.
- Pain reduction in the overstressed back muscles.
- Increase in mobility via a reduction in pain.
- Lower risk of falling due to an improvement of the statics of the body.

Patient instructions:


Step 1

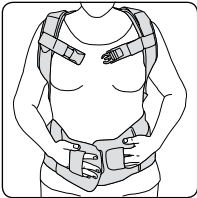The BORT DynamicFX Back brace that has been tailor-made for you and has been adapted by the orthopaedic technician is pulled over the shoulders in the same way as a backpack. In doing so, the base support is open. Please ensure that the shoulder straps are not twisted.


Step 2

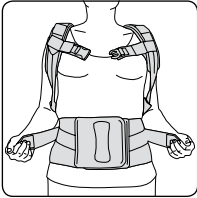Close the base support with as much tension as possible. In order to be able to close the base support with as little effort as possible, please use the hand loops that are attached to the closure plates of the base support. When closing, ensure that the base support is positioned above the groin and does not push in an unpleasant manner when sitting. Both of the base support fasteners must be connected to each other in a parallel manner.

Step 3

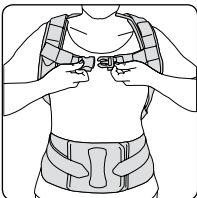Now pull both of the side bracing elements forwards in an even manner and secure them onto the base support.

Step 4

Then close your BORT DynamicFX Back brace chest strap.

Special information:

Please ensure that you have the fit and the functionality of the BORT DynamicFX Back brace regularly checked by your orthopaedic technician. It is especially important to inspect the length of the shoulder straps and the base plate and to have these components readjusted if necessary, particularly when starting treatment with the BORT DynamicFX Back brace.

In order to ensure safety when dealing with your medical aid, it is particularly important that you practice putting on the brace and taking it off together with your physician / physiotherapist / orthopaedic technician.

Wearing time:

The wearing time of your brace is to be determined by your treating physician depending upon the present indication and your personal situation within a treatment plan.

Please note that treatment with the BORT DynamicFX Back brace is a long-term treatment measure. The sustainable treatment success will only occur after a certain period of time has passed.

Here is an example of what the BORT DynamicFX Back brace wearing time **could** be like for the first 4 weeks:

	Week 1	Week 2	Week 3	Week 4
Mornings	1 hour	1,5 hours	2 hours	3 hours
Afternoons	1 hour	1,5 hours	2 hours	3 hours

Indications: Active straightening for relief and correction of the lumbar / thoracic spine associated with restricted movement in the sagittal plane, i.e. stable, osteoporotic vertebral fractures, osteoporosis, cautionary treatment of secondary kyphosis (stable fractures, tumors), muscular insufficiency, pre-operative and post-operative stabilization.

Contraindications: Inflammatory or injury-related changes to the skin (also includes swelling and redness) in the area of the body to be treated. Circulatory disorders, soft tissue swelling. Neurologically-related organoleptic and skin trophic disorders in the body area to be treated (sensitivity disorders, sensory disorders with / without damage to the skin). Prolonged immobilisation (contracture), particularly amongst the elderly.

Material composition:

Base support and shoulder padding:

Knitted section: 68 % Elastodiene, 32 % polyamide

Velcro: 100 % Polyamide

Velour straps/stomach plate: 70 % Polyamide Velcro velour, 30 % polyurethane

Shoulder padding: 35 % Polyamide, 35 % cotton, 30 % polyurethane ester

Back plate:

Metal back plate: AlMg 3

Belt straps, back covers, shoulder padding, spring covers: 74 % Polyamide Velcro velour, 26 % polyurethane

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

Technician information:

Adjustment of the BORT DynamicFX Back brace

The adjustment of the BORT DynamicFX Back brace may only be carried out by trained specialist staff.

Step 1


Determination of the circumference of the waist

Measure the circumference of the waist and select the appropriate size from the available base support sizes:

Base support sizes	Circumference of waist
Size 0	60 – 70 cm
Size 1	70 – 80 cm
Size 2	80 – 90 cm
Size 3	90 – 100 cm
Size 4	100 – 110 cm
Size 5	110 – 120 cm
Size 6	120 – 130 cm

Step 2

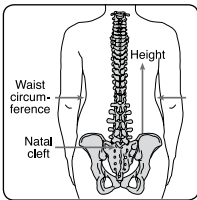Selection the base plate

The length of the spine is measured step by step via the spinous processes of the spine that run from the natal cleft up to the head (cranial).

The base plate is available in 4 different lengths:

Sizes	Base plate length
small	32 cm
medium	40 cm
large	48 cm
x-large	56 cm

The selection of the base plate is primarily determined by the affected area / areas as well as the individual body size of the patient.

Function of the spring tension elements:

The dynamics of the brace can be individually selected. The strength of the spring determines the degree of straightening and therefore, the improvement in posture.

The following spring strengths are available:

- 30 Newton
- 40 Newton
- 50 Newton
- 60 Newton
- 80 Newton

Spring strengths of between 80 N and 60 N are primarily used in **postoperative** cases. 50 N to 30 N springs should be used after half of the wearing time as recommended by the surgeon has passed.

Patients who have been treated in a conservative manner suffering from: muscular imbalance, adicular irritation syndrome, developing spinal stenosis, irritation of the sacroiliac joint, as well as those who are undergoing preventative stabilisation could benefit from spring strengths ranging between 30 N and 50 N. These strengths are always measured in relation to the individual constitution of the patient.

Patients who have been treated in a conservative manner suffering from: NPP (Nucleus Pulposus Prolapse), spinal stenosis, facet syndrome, osteochondrosis, spondylolisthesis are to be supplied with spring strengths ranging from 60 N to 80 N.

Strap lock

If dynamics from spring tension elements are not to be applied in the acute phase, in the conservative and postoperative phase (temporary), the **strap lock** can be placed in the two strap connectors of the individual straps.

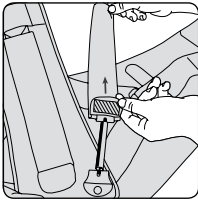
By integrating the **strap lock** in all three tension straps, the BORT DynamicFX Back Brace fixes the lumbar and thoracic spine when the abdominal section and shoulder straps are correctly closed.

Insertion / replacement of the spring tension elements:


All spring tension elements are labelled by colour based upon their spring constants:

30 Newton ➔ yellow 40 Newton ➔ white 50 Newton ➔ blue


60 Newton ➔ red 80 Newton ➔ black

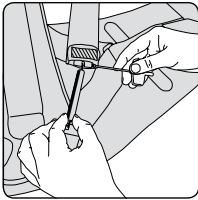To replace the spring elements, detach the press stud closure of the spring protective pockets and push them upwards.

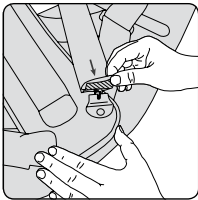
Open the Allen screws on both of the belt connectors using the supplied Allen key and replace the spring tension elements if required. Re-tighten the Allen screw afterwards.

Now pull the spring protection pockets back down and reclose the press stud closure on the base back plate.

All 3 belts of the BORT DynamicFX Back brace should be equipped with the same spring strength.

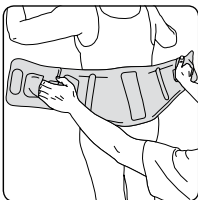
Applying the body section:

Before applying the base support, remove it from the base plate by detaching the Velcro. Position the base support together with the bottom edge of the back section level with the natal cleft.

The patient now closes the base support at the front.

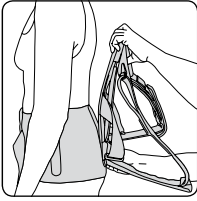Adjustment of the base plate:


Now adjust the base plate to correspond to the patient's spine and to match the contours of the body. **The base plate can be cold-formed without tools.**

When dealing with patients who are suffering from severe inclination positions, ensure that the spine is ideally straightened.

Now secure the base plate onto the base support. Ensure that the base plate is form-fitted with the base support on the bottom edge.

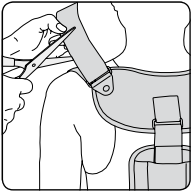Adjustment of the reclination and shoulder straps:

Shorten the reclination and shoulder straps in the following order:

Step 1

Shorten the middle belt in such a manner that the upper edge of the shoulder plate ends approximately 5 - 10 cm below the C7 cervical vertebrae.

Step 2

Then shorten both of the shoulder straps evenly.

In order to shorten the shoulder straps, please remove the crocodile fasteners that are located on the ends of the straps respectively. After shortening, secure the crocodile fasteners again and re-tighten the brace straps.

Ensure that the spring elements are slack when the body is in a reclined position. When the patient takes a neutral posture, a looser torsion strength of the shoulder straps increases the wear comfort, yet retains the therapeutic function.

Adjustment and application of the chest strap:

In order to shorten the chest strap, please remove the crocodile fasteners that are located on the ends of the straps respectively. After shortening, secure the crocodile fasteners again and re-fasten the brace straps onto the shoulder straps.

Shorten the chest strap in such a manner that the shoulder straps do not move into the armpit area.

The shoulder straps can be moved forwards by lightly pulling the chest strap. This prevents rubbing in the armpit.

The chest strap can be easily opened and closed via the quick release fastener located in the centre of the strap.

Secure the supplied shoulder padding underneath the shoulder straps if necessary.

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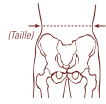
Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Close the Velcro closure to avoid damaging other items being washed. Shape by hand and allow dry in the open air. Avoid heat.

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The length of the base plate is determined by the required stabilisation level.

Please ensure that you always order the base support and the base plate together (e.g. Size 3/medium).

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BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com