# SellaFlex Sport

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Thumb support | Soutien du pouce | Órtesis de pulgar |  
Supporto per il pollice

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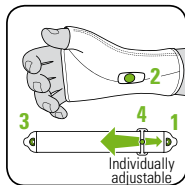
The flexible **wrist support with the SellaFlex Sport thumb support from BORT** serves to stabilise the metacarpophalangeal joint and saddle joint of the thumb. Here, the compression strengthens the retaining function of the ligaments, thus relieving the joint. The thumb hollow hinders the support from slipping. Thanks to the highly elastic SOFTFlex 3D knitted fabric, the support is easy to put on and nonetheless achieves a good supporting effect.

**COOLMAX®** fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

### Instructions for use:

**Important:** When you receive your thumb support, the supporting module for use on the right hand is inserted. The attachment **Stabilo®** strap module **(1)** is attached to the support in the supporting module **(2)**.



Place the support on the metacarpophalangeal joint affected and guide the thumb through the intended thumb opening at the same time.

Stabilo® strap with the easyClick system for additional stabilisation of the wrist:


- 1** and **3** attachment module
- 2** supporting module
- 4** adjustment module

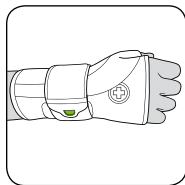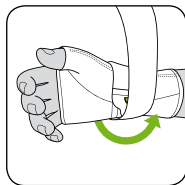Adapt the stabilisation degree of the Stabilo® strap by moving the adjustment module **(4)** individually.

Guide the Stabilo® strap circularly around the wrist under slight tension.

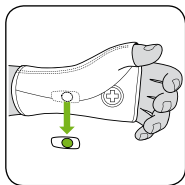Close the strap by attaching the free attachment module **(3)** to the adjustment module **(4)**. Pay attention to comfortable fitting of the Stabilo® strap, neither too loose nor too tight.

### **Left hand thumb support**

If the thumb support with the Stabilo® strap is needed for use on the left hand, hang the Stabilo® strap out of the support. Remove the supporting module (2) from the velour bag and slide it into the velour bag on the opposite side. The supporting module should always be located on the lower side of the arm towards the palm.

If the Stabilo® strap with the easyClick system is not needed, the supporting module (2) can be taken from the velour bag.

### **Indications:**

Chronic, post-traumatic or post-operative irritations in the area of saddle joint of the thumb, e.g. in the case of ulnar and radial ligament instability, early stages of rhizarthrosis

### **Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

### **Composition:**

40 % polyamide (COOLMAX®), 40 % polyamide, 19 % elastane (LYCRA®), 1 % silicone

### **Important Note:**

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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Hand wash support with warm water up to max. 30 °C with mild or mild detergent. Fix the Velcro closure to avoid damage to other laundry. Pull into shape and air-dry. Avoid heat.

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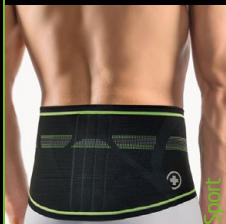
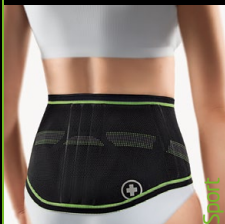
**EN** The **BORT SellaFlex Sport thumb support** stabilises the metacarpophalangeal joint and saddle joint of the thumb, thus providing support in load situations. The **SOFTflex 3D knitted fabric** and moisture regulating properties of the high-tech **COOLMAX® fibre** ensure that it is very comfortable to wear and for ideal properties for sports activities. The **Stabilo® strap** with the innovative **easyClick system** stabilises the wrist additionally.

**BORT Quality products** promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | [www.bort.com](http://www.bort.com)

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