**de** MucoClear® 6%

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**Stand der Information:** September 2011

**en** MucoClear® 6%

**Content:** 4 ml hypertonic sodium chloride inhalation solution (NaCl 6%) - sterile

**Area of use:** Increases the mobilisation of secretions in the lower respiratory tract in patients with persistent mucus accumulation (e.g. cystic fibrosis, bronchiectasis) because of its osmotic effects.

**Type of application:** with PARI nozzle nebuliser or eFlow®rapid.

**Dosage:** 4 ml to be taken 2 to 4 times daily.

**Warnings:**

-  – Follow the appliance manufacturer's instructions for use.
- If the patient displays a tendency to develop dyspnea or hypersensitivity, this product must not be used except under medical supervision.
- Do not mix with other medications!
- MucoClear 6% contains no preservatives. Therefore, opened ampoules must be disposed of immediately for reasons of hygiene.
- Discard this product after its expiration date!
- This product includes small parts, which could pose a choking hazard. Always keep this product out of the reach of children aged three or younger!

In rare cases, temporary irritations (for example, coughing) or reversible constriction of the bronchia may occur. Doctors often recommend that a bronchodilator be used before treatment. If undesirable side effects occur, discontinue treatment and seek advice from your doctor or pharmacist.

**Dosage form/package sizes:** 20 x 4 ml and 60 x 4 ml

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