Norditalia

IT.....	MANUALE D'USO	4
EN.....	OPERATING MANUAL	11
RO.....	MANUAL DE UTILIZARE	17
FR.....	MANUEL D'UTILISATION	23
PL.....	INSTRUKCJA OBSŁUGI	29
NL.....	HANDLEIDING VOOR DE GEBRUIKER.....	35
DE.....	GEBRAUCHSANWEISUNGSHEFT	41
ES.....	MANUAL DE USO	47
CZ.....	NÁVOD K POUŽITÍ	53
SK.....	NÁVOD NA POUŽITIE	59
AR.....	دليل الاستخدام	70

MO-03

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

⚠ IMPORTANT INDICATIONS FOR PROPER USE

Some medicines are not suitable for electronic, ultrasound or mesh nebulizer use. Always consult how a medicine should be used and your doctor. Unsuitable medicines may result in longer aerosol drug delivery time or failed delivery of the same.
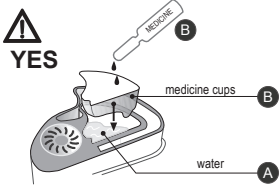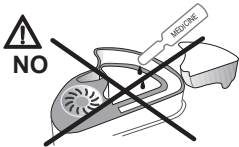
<p>Always use medicine cups</p>  <p>Make sure the tank is always filled with water</p>		 <p>The medicine cup must be replaced when:</p> <ul style="list-style-type: none"> - it is damaged or holed - it has been used for 5 treatments <p>Do not use holed cups.</p>
--	---	---

Rechargeable battery (optional)

Do not leave the battery on charge for more than twelve hours.

Overcharging can bring the risk of shortening the life of the battery.

Avoid discharging the battery excessively. Recharge at least every three months.

Always remove the battery from the nebulizer when not in use. Leaving the battery in the appliance by mistake flattens the battery.

INTENDED USE

The ultrasound aerosoltherapy device is a medical device used in combination with medicines and destined to be used on human beings for a therapy or lessening of a respiratory disease. The medicine to be used, the correct amount and delivery method have to be prescribed by a doctor. The medicine delivery is performed through nebulization of the medicine in a suspension of solid particles or liquid drops which penetrate in the airways through nose or mouth and are absorbed by the patient.

There are currently no contraindications regarding the use of the medical device. The main side effects may be caused by the medicine used.

PATIENTS AND USERS

The device is intended for patients suffering from respiratory problems to which a doctor has prescribed the use.

The user could be the patient himself or, if the patient isn't able to perform the therapy independently, the one who assists the patient.

The patient and the user must fully understand the contents of the instruction manual and be able to carry out the necessary operations for the correct use of the device following the doctor's prescription.

PRINCIPLE OF OPERATION OF THE APPLIANCE

The device consists of a nebulizer composed of a piezoelectric crystal that creates a vibration so intense that turns the medication into very fine particles. Through the use of the appropriate accessories, these particles introduced inside the respiratory system of the patient.

ESSENTIAL PERFORMANCE

The nebulized quantity must be greater or equal than to 50% of the declared value. The test should be run at maximum speed using 4 ml of saline solution.

RESPONSIBILITY

The manufacturer, his representative, or the importer shall only be considered responsible in terms of safety, performance and reliability if:

- the assembly technique, calibration, repairs or amendments have been performed by persons authorised by them;
 - the electrical system complies with the standards in force;
 - the operating instructions have been respected;
- The manufacturer declines all responsibility for damage deriving from improper, erroneous and unreasonable use. By the time the product may deteriorate. After 5 years have passed from the manufacturing date stated on the data plate label the product is no more usable and the manufacturer cannot be held responsible for damages or incidents caused by the product.

IMPORTANT WARNINGS

⚠ Failure to follow the warnings and the content of the present manual can expose to heavy risks such as: health state worsening, scald, fire, electrocution, choking, cross-infections, re-infection, death, environmental contamination.

Failure to comply with the instructions and the contents of this manual may result in reduced performance of the appliance, malfunctioning and compromised safety.

- Read all the instructions carefully before using the appliance.
- Respect the electrical specifications indicated on the appliance; If necessary, use adapters or extension leads that are compliant with the safety standards in force, paying attention not to exceed the rated current value and the maximum power value indicated on the adapter.
- If the body of the appliance or its power cord are damaged, do not use the appliance.
- The inhalation of any substance must be prescribed and monitored by the doctor, who will indicate the most suitable doses and treatment methods. The therapeutic effectiveness of the treatment must always be verified by the doctor;
- Some medicines are not suitable for electronic, ultrasound or mesh nebulizer use. Always consult how a medicine should be used and your doctor.
- Electricity supply failure, an unexpected fault or any other unfavorable condition may cause the appliance not to work; we therefore

- recommend that you have an appliance or medicine (subject to agreement with your doctor) that may be used as an alternative;
- Only use the appliance for the use foreseen (aerosol inhalations) and as described in this manual;
- Do not let children or incapable persons use the appliance without supervision; in such cases, pay special attention during treatment;
- Before use, check the integrity of all the components present in the pack; if they are damaged they must not be used;
- Never leave the appliance and all its parts, including packaging, within the reach of children, incapable people, domestic animals or insects.
- The elastic bands (optional) can include latex (LATEX). Latex can produce allergic reactions. 
- Do not use the appliance with wet or damp hands or while taking a bath; Do not wet the appliance with water or other liquids; do not use the appliance if it has accidentally become wet;
- Keep the appliance and the power supply cable away from sources of heat; The appliance is not suitable for use in the presence of inflammable substances;
- Do not use the appliance in rooms where sprays have just been used; air the room well before proceeding with the therapy;
- Do not block or obstruct the appliance air openings and do not place it on a soft surface such as a bed or settee;
- Do not insert any objects into the cooling openings; Do not use the appliance in cases where it makes unusual noises;
- To guarantee the performances, functionalities and safety of the appliance, only use Norditalia original accessories and spare parts.
- Only and exclusively use the adapter and the rechargeable battery supplied with the appliance;
- We recommend personal use of the accessories; In order to reduce the risk of infection, we recommend to sterilize the accessories before each use.
- Do not pull the power supply cable or the appliance to remove the plug from the power supply socket. Always remove the plug after use. In case of detachable cable always remove the plug from the wall socket before detaching the cable from the appliance.
- The nebulizer is designed to operate for 15 minutes (on) / 30 minutes (off);
- Do not leave the appliance exposed to atmospheric agents; store it in the environmental conditions indicated.
- Do not wrap the tube or the power cable around your neck.
- The appliance include some small parts which can be dangerous if inhaled or swallowed.
- Do not operate the device if stored in dirty places.
- Always switch the nebulizer off after use. Always remove the adapter from the current socket or remove the rechargeable battery from the nebulizer;
- Do not pour liquids other than normal tap water into the tank.
- Do not insert medicine into the water tank.
- Do not switch the machine on with the lid open. Do not open the appliance during operation.
- Do not approach the medicine outlet to your eyes.
- During treatment do not occlude the air inlet or move the appliance awkwardly.
- Do not use the appliance by connecting it to other appliances and accessories that are not described in the instructions.
- Do not use the appliance outside the intended environmental conditions of use.
- Before using the appliance, keep it at the environmental conditions of use for at least 24 hours.
- It is forbidden to tamper with the device and any of its parts.
- Any repairs and maintenance must be carried out exclusively by the manufacturer or by authorized personnel

OPERATING INSTRUCTIONS

Very important: do not switch the appliance on before performing all the operations indicated in the manual.

In particular, do not switch it on without placing water in the tank as it could be seriously damaged.

1. If the nebulizer has never been used, clean as described in the "CLEANING PROCEDURE" section.
2. Open the appliance cover.
3. Keeping the adapter detached from the electricity supply, insert the adapter output plug into the socket in the lower part of the appliance (fig. 2).
4. Position the appliance on a stable surface in a way that the adapter is not crushed between the appliance and the surface (fig. 3). If using the base, act so that the cable remains in the correct position (fig. 4).
5. Fill the tank with tap water up to the mark indicated (fig. 5). We recommend filling with fresh water each time. The nebulizer does not work without water. If the tank is not filled correctly, the appliance will not work and the signal  appears on the display
6. Position the medicine cup on the tank filled with water (fig. 6)
7. Insert the prescribed medicine into the cup (fig. 7). For this operation avoid using sharp parts (like a syringe needle for example) that could cut or hole the cup.
8. Do not pour in more medicine than the cup can hold, equal to 8 ml. The maximum capacity is indicated by the reference mark on the medicine cup (fig. 8). If the quantity of medicine to inhale is greater than 8 ml., divide the dose into several parts. Dense medicines or quantities of less than 2 ml. must be diluted with at least 2 ml. of physiological solution to obtain the minimum volume of 4 ml.
9. Replace the cover on the nebulizer body.
10. Connect the adapter to the power supply after checking that the power supply voltage corresponds to the rated voltage indicated on the adapter itself (fig. 9). Correct connection is indicated by the green Led coming on.
11. Connect one end of the extending tube to the cover (fig. 10).
12. Connect the other end of the tube using the accessories connector (21), if provided, to one of the accessories as indicated in the accessory connection diagram (fig. 11). Pay attention that the weight of the accessory used (the adult mask, for example) does not create an imbalance causing the appliance to fall.
13. When the appliance is new it is normal for the appliance and accessories to have a peculiar smell which leaves after few uses.
14. Press the  button once to switch on the appliance. The appliance can operate at 3 different speeds, displayed on the screen by these symbols: Maximum speed: . Medium speed . Minimum speed .
15. The emission of nebulized medicine confirms correct functioning.
16. To regulate the air-flow in a different way, (medium and minimum delivery) press the  button until the desired speed is reached.

17. During operation, symbols may appear on the screen. Refer to the "Signal summary table" for their specific meanings.
18. Inhale the nebulized medicine until exhaustion. It is normal for a certain amount of medicine (approximately 1 ml.) not to be nebulized. The not nebulizing quantity, such as the nebulization speed, depend on the medicine characteristics. For example, drugs highly viscous or producing foam may be more difficult to nebulize.

19. At end of the treatment press the  button once to switch off the appliance.
20. Paying attention to not spilling the liquids contained in the appliance, disconnect the appliance power supply by disconnecting the electrical power supply of the battery from the appliance.
21. The green Led must be off.
22. Clean the appliance as indicated in the "CLEANING PROCEDURE" section.

CLEANING PROCEDURE

 **All the cleaning operations must be performed keeping the nebulizer disconnected from the electrical power supply.** Disconnect the adapter from the power supply socket. Disconnect the adapter pin from the lower part of the appliance, taking care not to let liquid come out of the appliance. If used, disconnect the rechargeable battery. The green Led must be off. During cleaning activities, prevent the electrical parts, in particular those in the lower part of the nebulizer, from coming into contact with liquids.

Raise and remove the cover. Extract the medicine cup and empty it of any remaining medicine. There is normally a small quantity of medicine that is not nebulized. Completely empty the water contained in the tank.

The cover, cup, masks and mouthpiece must be washed delicately with tepid water and successively dried with a clean cloth.

Clean the nebulizer body and the tank with moistened damp cloth (fig. 12). Do not submerge the appliance in water. If present, eliminate any lime scale deposits from the piezoelectric ceramic (fig.13) and from the metal water sensor (fig. 14) using small stick with cotton wool on the end.

 **For these operations, do not use sharp or abrasive objects and take care not to exert excessive force that could cause breakage.**

If encrustation is found between the fan and the appliance, wet with water. For this operation we recommend using a syringe without a needle (fig. 15). Do not force or bend the fan.

If necessary, remove lime encrustation from the piezoelectric ceramic located in the tank by inserting 1 part of water and 2 parts of white vinegar. Let it act for several hours and rinse several times so as to eliminate vinegar residues. Dry with a soft cloth.

ACCESSORY CLEANING

After use, and if necessary, before use, wash the accessories carefully with tepid water and dry them with a cloth. Then put the accessories into a clean storage area. When cleaning, ensure that all residue is removed and that the accessories are dried properly. Under no circumstances should you use substances that may be toxic if they come into contact with the skin or mucous, if ingested or inhaled.

If it is necessary to disinfect the accessories, adopt the methods approved by the regulations in force and in compliance with the resistance of the material (see Table 1: List of material). The accessories may be disinfected using specific products such as the MILTON® method: they should be rinsed to remove all traces of disinfectant, dried and stored in a dry and clean environment.

The accessories may be disinfected in boiling water (100°C) for at least 40' or in an autoclave (120°C) for at least 20'. Do not boil the masks.

Do not put weights on the accessories and do not bend them; do not exceed the temperatures indicated; After disinfection, store in clean bags. PVC accessories will lose their transparency in water or vapour; this phenomenon will disappear after a few days in a dry environment and will not affect usability. All the accessories may be disinfected up to a maximum of 25 times, following which they should be replaced.

The frequency and interval between one disinfection session and another should be established according to the discretion of the doctor and/or the clinic and according to the pathology. The accessories should normally be disinfected prior to each change of patient. Do not wash accessories in a washing machine.

In case of use onto high-risk groups (e.g. cystic fibrosis patients), consult your doctor who will prescribe specific precautions.

REPLACING THE MEDICINE CUP

The medicine cup must be used when:

- it is damaged or holed; **do not use holed cups;** 
- it has been used for 5 treatments;
- the user experiences a clear fall in the performance of the nebulizer;
- a medicine different to the one used previously has been used;
- the nebulizer is used on a new patient.

The ribbing present on the medicine cup is a production characteristic and not a production defect.

REPLACING THE FILTER

The filter prevents external elements from blocking the appliance fan. The filter must be replaced when excessively dirty or when air can no longer pass through. If the filter is encrusted with medicine, it can be rinsed with tepid water and then dried with a clean cloth (fig. 16).

RECHARGEABLE BATTERY (OPTIONAL)

 **Important warnings:**

- The rechargeable battery (optional) is used to supply the appliance as an alternative to the adapter. For this purpose, only and exclusively use the rechargeable battery supplied.
- The rechargeable battery must not be used for other appliances or for purposes different to those indicated in this manual.
- Do not expose the battery to high levels of humidity or high temperatures (e.g. sources of heat, direct sunlight, etc.);
- Do not place the battery on wet surfaces.
- Avoid wetting the rechargeable battery. In particular, pay attention to not inserting liquids and/or sharp objects into the power supply sockets located in the upper part.
- In the case of incorrect use or conservation (battery kept completely submerged in water or kept in close contact with heat sources) the

- battery may overheat and in extreme cases explode.
- The battery can be recharged up to around 300 times. Its life is anyway variable, depending on how it is used, conserved and charged.
- Only and exclusively use the adapter provided for battery charging.
- It is normal for the battery to become warm during charging and use.
- High or low temperatures limit battery operation.
- The battery can be damaged if left completely depleted for a long time. Recharge at least every three months.
- Battery charge is constantly monitored. When the flashing battery symbol lights up, the device will turn off after about 2 minutes. If the battery level is too low, the flashing symbol of lack of water lights up and the appliance will turn off immediately.
- Do not open and tamper with the battery pack in any way.
- Do not place the battery pack in direct contact with fire.
- Do not short-circuit the battery contacts.
- Do not use the battery if it has been damaged.

BATTERY CHARGING

Connect the adapter to the current socket. Connect the adapter to the power supply socket of the rechargeable battery located next to the symbol  (fig. 20). If the battery is provided with a led, the blue led will blink during the charge. When the battery will be fully charged, the led will turn off.

 **Do not leave the battery on charge for more than twelve hours.** Overcharging can bring the risk of shortening the life of the battery. Avoid discharging the battery excessively. If the appliance signals a flat battery by the symbol  coming on, switch it off immediately. Recharge as soon as possible. A completely new and charged battery has around 45 minutes of autonomy. It is normal for the battery to loose performance as time goes by.

BATTERY USE

Place the rechargeable battery on a flat surface. Delicately rest the battery on the nebulizer body so that the two parts couple correctly (fig. 21). Correct coupling is indicated by the Led coming on (19). If the battery pack does not go in correctly, do not force it: repeat the operation delicately. Use the appliance as described in the "use instructions" section.

At the end of the treatment delicately remove the battery from the nebulizer, avoiding spilling the liquids in it.  **Always remove the battery from the nebulizer when not in use.** Leaving the battery in the appliance by mistake flattens the battery.

USING THE TABLE BASE

For better stability and convenient use, the appliance can be placed on the table base with three feet that extend (fig. 17). The support base has an attachment belt for fitting the appliance to the sides of a bed or other supports. Slide the fixing belt into the base opening so that the non-slip rubber part locates on the opposite side to the device (fig. 18). Position the base on the fixing element. Adhere the non-slip rubber to the fixing support and firmly close the belt with the Velcro® (fig. 19).

EXPECTED LIFE

The intended life of the appliance and its accessories shall be as follows:

- Device 5 years (15 minutes a day).
- Accessories (Snorkel, Mask, Tube, Accessories joint) 12 months.

The life of the accessories may vary depending on the environment and the conditions of use. Frequent use of the product can shorten the expected life.

PERIODIC CHECKS

 Damaged accessories should be replaced. Although a correctly used appliance can last a very long time, we recommend to replace it after no more than 5 years from the manufacturing date. Periodically, or in case of doubt about the performances of the appliance, verify for the "Amount nebulized" not to decrease below 50% of declared value. The test have to be performed using 4 ml of physiological solution at maximum speed. If the performances would be not compliant try replacing the filter, repeat the cleaning procedure or get another appliance.

WASTE DISPOSAL

 Dispose of the appliance and its accessories in compliance with the regulation in force. Do not dispose of this appliance in the environment. In case of doubt please call your local authorities.

GUARANTEE CONDITIONS

This appliance is guaranteed for 2 years, except where otherwise indicated or for different local regulations. The guarantee is only valid if the device is returned to the stockist, together with its receipt as proof of the date of purchase. The guarantee covers the appliance for any fabrication defects, on condition that the same has not been tampered with by the user or by persons other than the authorised maintenance service personnel. For service or for reporting abnormal operation please contact the manufacturer or your local distributor. The guarantee does not include:

- faults or defects caused by improper use, or failure to comply with the operating instructions;
- interventions for presumed defects, checks on request, ordinary maintenance;
- accidental damage such as shocks, falls, etc.
- Components subject to wear and tear (battery, accessories, etc...).

NB. For repairs or overhauls, return the appliance to the stockist in clean condition.


ELECTROMAGNETIC COMPATIBILITY

The appliance is compliant with the electromagnetic compatibility standards currently in force and is suitable for use in all buildings, including those for residential / domestic use. The RF emissions from the appliance are very low and are not likely to cause any interference with nearby appliances. In any case, we recommend that you avoid positioning it on top of or near other appliances. If it does appear to be interfering with your electrical appliances, move it or connect it to a different electrical socket. Radio appliances may affect the functioning of the appliance, keep them at least 3 m. away. For more information, please visit www.norditalia.biz.

APPLIANCE TECHNICAL DATA**

Power: approx 30VA ; Amount nebulized: >1.0 ml/min (maximum speed); Amount inhaled (A.O.R. EN 13544-1): 0.4 ml/min; Aerosol Output Rate (EN 13544-1): 0.24 ml; Medicine cup capacity: 8 ml.; Noise level: < 15dBA; Operating times: 15' (on) / 30' (off).

ADAPTER ELECTRICAL DATA

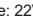Input voltage: please refer to the data label on the adapter; Output voltage: 22V  0,6A


RECHARGEABLE BATTERY TECHNICAL DATA

Battery type: Ni-MH; Electrical data: 21,6V  700 mAh

LEGEND (FIG 1)

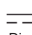- | | | | |
|-------------------------------|-------------------------------------|---------------------|------------------------------------|
| 1. Filter cover | 7. Water tank | 13. Extending feet | 19. Power supply LED |
| 2. Filter (1 + 2 spares) | 8. Appliance body | 14. Paediatric mask | 20. Charge Led (Blue) - (optional) |
| 3. Fitting for extension tube | 9. Keyboard | 15. Adult mask | 21. Accessories joint*** |
| 4. Cover | 10. Rechargeable battery (optional) | 16. Mouthpiece | A water |
| 5. Medicine cups (10 pieces) | 11. Belt | 17. Extending tube | B medicine |
| 6. Fan | 12. Table base | 18. Adapter | |

***Accessories joint (21) is not supplied if masks are already equipped with joint

Environmental Conditions for use / storage		Temperature 5 + 40 °C / -25 + 70 °C	Relative Humidity 15 + 90% / < 90%	Atmospheric Pressure 700 + 1060 hPa		
 Class II appliance	 Type BF appliance	 Important warning	 Consult the instruction manual	 Alternated current	ON OFF On/Off	IP22 Input Protection of: - objects --> minimum size 12.5 mm. - Liquids --> Dripping water when tilted at 15°
 CE 0051 CE marking	 Direct current	 Temperature Limits	 Humidity Limitation	 Keep dry	 SN	 Manufacturer

* For the models sold outside of the European Union, please refer to the data label on the appliance.

** Data variable on the basis of the medicine used. The measurements have been taken with the use of laser diffraction technology using a physiological solution. The data supplied may not be applicable to suspension or very viscose medicine, consult the drug manufacturer.

All the technical data in the operating manual are indicative.

PROBLEM	DISPLAY INDICATION	CAUSE	SOLUTIONS
The appliance switches itself off or does not come on.		No water or water below the reference level	Fill the reservoir suitably
		Excessively low battery	Charging the battery
When using the appliance the battery symbol flashes		The battery is being charged	Switch of the appliance and charge the battery
		If you are using the adapter it indicates a temporary lowering of the power supply voltage	You can continue with the therapy, unless the nebulization becomes insufficient
The appliance does not come on and the battery symbol flashes		Flat battery	Charge the battery or power the appliance with the adapter
The appliance switches itself off after around 15 minutes. The timer symbol comes on and stays on		Timer intervention	Let the appliance cool for at least 30 minutes
The appliance switches itself off and the timer symbol flash		The appliance is overheated	Let the nebulizer cool for at least 30 minutes
The appliance remains switched off after pressing ON/OFF	Green LED switched off	Adapter not connected correctly	Check that the adapter is well connected both to the nebulizer and to the electrical socket
		Battery not connected correctly	Make correct connection. See the use manual
		Appliance/battery / adapter damaged	Contact the stockist or an assistance centre
The appliance remains switched off after pressing ON/OFF	Green LED switched off + blue LED off while charging the battery	Battery / adapter damaged	Contact the stockist or an assistance centre
Nebulization very poor or absent		Too much medicine (more than 8 ml.) in the medicine cup	As indicated in the "USE INSTRUCTIONS" subdivide the dose
		Small amount of medicine	As indicated in the "use instructions" do not insert less than 4 ml.
		Medicine not inserted	Check that medicine is inserted in the medicine cup
		Medicine not suitable for use with electronic, ultrasound, mesh appliances	Check the instructions enclosed to the medicine and consult a doctor
		Small amount of water in the reservoir	Check that the water arrives at the level indicated
		Fan blocked by encrustation	See cleaning instructions
		Wet filter / dirty filter	Rinse the filter with water and dry it with a cloth <i>if necessary</i> replace the filter
Noisy appliance		Noisy fan	Check that the fan is not encrusted with medicine. Check that the filter does not touch the fan.
Sudden nebulization interruption		Air bubble between the cover and the tube	Blow gently into the tube
		Wet filter	Rinse the filter with water and dry it with a cloth
		Medicine has escaped from the medicine cup due to brusque movements	Replace the medicine

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