Spermix Privat Seminal Colection Device

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Simple recovery of semen in a non-latex medical
condom for testing and treatment in fertillity or
infertillity testing.

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Spermix Privat

Seminal Collection Device

Simple recovery of semen in a non Latex and non spermicidal medical condom for testing and treatment in fertility or infertility testing.

Instruction for use

The genital area should be cleaned before the spermix private is applied (e.g. with femsoft Intim Washing Lotion). The spermix private is then used in the manner of conventional condoms. If you use the non spermicid lubricating gel at the outer side of the Spermix Private, the intercourse is as normal as without. After ejaculation, when the sample in the tip of the medesign condom, carefully remove the condom from the flaccid penis, leaving the ejaculated specimen is in the tip of the Spermix Private. Twist it several times about the middle and apply the twist-tie included in this package. Return the Spermix Private to the zip-lock bag and close.

Note the time of specimen collection, patient's name, and physician's name. The bag containing the sample should be returned to the physician or laboratory as soon as possible to insure the most accurate result.

Note:

To protect the integrity of the spermatozoa, please maintain the specimen at body temperature during transport. This can be accomplished by placing the specimen flat against the body (under garment) in the chest or abdominal area.

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medesign I.C. GmbH · Dietramszellerstr. 6 · D-83623 Dietramszell