# Rheunerval® M Creme

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1141114/17

# Rheunervol<sup>®</sup> M Cream

Medical device to support massage for muscular and articular rheumatism, muscular tension, lumbago, low back pain

## INDICATIONS FOR USE

### 1. THERAPEUTIC PURPOSE

*Rheunervol<sup>®</sup> M Cream* applied to uninjured skin as lubricant for massages for physical treatment.

### 2. THERAPEUTIC INDICATIONS

*Rheunervol<sup>®</sup> M Cream* is applied for massages that are applied for the therapeutic treatment of muscular and articular rheumatism, muscular tension, lumbago or low back pain.

The product can be applied both by adults at home or in the professional treatment of patients, the disabled and elderly.

### 3. UNDESIRABLE EFFECTS

When applying *Rheunervol<sup>®</sup> M Cream* local irritations or contact eczema may occur.

A temporary flare in the range of application, however, only indicates increased circulation as intended in the massaged area.

### 4. HISTOCOMPATIBILITY

No data is available that suggests there may be a histoincompatibility to the product.

### 5. CONTRAINDICATIONS

#### Attention:

The product may not be applied to acutely inflamed articulations and injured skin, or to the face.

### 6. GENERAL SPECIAL PRECAUTIONS

*Rheunervol<sup>®</sup> M Cream* may not come into contact with eyes, mucous membranes or open wounds.

#### Attention:

*Rheunervol<sup>®</sup> M Cream* may exclusively be applied by adults. The application to nurslings and children under the age of 16 and to pregnant and breast feeding women is excluded.

#### Attention:

The product is solely designed for external application to uninjured skin!

#### Attention:

Wash hands thoroughly with water and soap after application.

#### Attention:

Keep out of the reach and sight of children!

#### Attention:

*Rheunervol<sup>®</sup> M Cream* may not be applied when hypersensitivity to one of the ingredients is known.

### 7. PRODUCT PROPERTIES

With *Rheunervol<sup>®</sup> M Cream* the skin becomes well lubricated and smooth. The product regulates transmission of power to the muscles in the massage movement and thus prevents overstraining.

The lubricating properties of the product supports the flowing and effective application of the kneading, rubbing and stroking grips. A deeper relaxation and loosening of the muscles is made possible.

The product contains low concentrations of camphor as odorant substance.

### 8. PRODUCT APPLICATION

Massage is a widespread form of physical pain treatment of rheumatic morphology, myalgia, neuralgia and articulation complaints which can be applied oneself or in the form of a professional therapy.

*Rheunervol<sup>®</sup> M Cream* is applied at lengths of 2-4 cm for the treatment as required, applied in a thin layer to the region to be massaged and then rubbed in gently.

The product is intended for the long-term application until the symptoms subside.

The used up tube and folding box is disposed of in domestic waste, preferably separated.

### 9. PHYSICAL PRODUCT DATA

Composition:

2-Propanol, Propylene glycol, Isopropyl myristate, Cetomacrogol 1000, viscose paraffin, polyacrylic acid, sodium hydroxide, purified water, 1.5 % racemic camphor as odorant substance

Appearance, odour:

Light, yellowish cream with slight odour of camphor.

Storage conditions:

*Rheunervol<sup>®</sup> M Cream* is stored at room temperature (not over 25°C).

Close tube tightly after use.

Shelf-life: 2 years

### 10. DELIVERY SPECIFICATIONS

Tube with 100 g cream, packed in folding box

### 11. MANUFACTURER / DISTRIBUTOR

Manufacturer and distributor of the medical device:

 Abanta Pharma GmbH  
Reichsstraße 78  
58840 Plettenberg  
Telephone: + 49 341 2582 190  
Telefax: + 49 341 2582 191  
E-Mail: info@abanta-pharma.de

 Manufacturer

 Temperature limits

 Shelf life

 Batch definition/Batch number

 Note Instructions for Use

#### Note:

Please read Indications for use.

This leaflet was last approved in October 2017.

1141114/17

 Abanta  
Pharma