# VisuEVO®

OPHTHALMIC SOLUTION

# VisuEVO®

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## EN Instructions for use

VisuEVO® is an innovative formulation containing omega-3 essential fatty acids, Vitamins A and D and ultra-filtered phospholipids. VisuEVO® reduces excessive evaporation of the tear film, which is caused by inflammatory conditions on the surface of the eye, due to the presence of liposomes (phospholipids), Omega-3 and Vitamin D.

Vitamin A and phospholipids are natural constituents of the tear film. Their antioxidant action prevents damage caused by reactive oxygen species and facilitates homeostasis and the retention of the tear film, as well as the control of its evaporation.

Omega-3, Vitamin D and Phospholipids improve the osmolarity of the tear film, protecting it from inflammatory conditions caused by excessive evaporation.

Due to its formulation, VisuEVO® is indicated:

- for all forms of evaporative dry eye, to improve the stability of the lipid layer by reducing the evaporation of the aqueous phase, i.e. the tear film (which can be caused, for example, by prolonged use of computers, reading, driving, the use of medicines, ocular allergies and contact lenses);
- as a pre-treatment for eye surgery, in order to help with the restoration of the lipid integrity on the surface of the eye;
- for forms of dry eye caused by a reduced secretion of lipids from the Meibomian glands;
- for cases of dry eye caused by a mucin deficiency;
- for cases in which dry eye has worsened as a result of the use of preserved eye drops.

### HOW TO USE:

Instil 1 drop of VisuEVO® into the conjunctival sac of each eye 3 times a day or according to the different instructions of your ophthalmologist.

### HOW TO APPLY VISUEVO®:

Hold the bottle just under the cap and turn the cap to open it.

Instil 1 drop into the conjunctival sac of each eye by gently pulling down the lower eyelid and looking upwards. Dispense the product by gently squeezing the bottle.

Close the bottle immediately after use. The solution can continue to be used for 60 days after the first opening.

### INSTRUCTIONS FOR USE:

If you are using other eye drops at the same time, it is recommended that you wait at least 10 minutes after administering other medications before administering VisuEVO®.

### WARNINGS AND PRECAUTIONS:

- Do not use the product if you are allergic or hypersensitive to any of its components;
- If you have any adverse reactions, discontinue the use of the product and consult your ophthalmologist;
- Close the bottle well after use;
- Do not use the product after the expiry date indicated on the package;
- The product remains sterile for 60 days after first opening of the bottle;
- Wash your hands before applying the product;
- Do not touch the eye with the tip of the bottle;
- Do not use the product if it has been damaged;
- The expiry date indicated applies to the unopened, correctly stored product;
- Keep the product away from sources of light and heat and out of the reach of children;
- The product can be applied while wearing contact lenses;
- For ophthalmic use, do not swallow;
- Store the product at room temperature (15-25°C).

### COMPOSITION:

Liposomes from soya phospholipids, algae oil (DHA+EPA), Vitamin A palmitate, Vitamin D, disodium EDTA, PEG 400, Boric acid, Sodium tetraborate, buffered isotonic buffered with pH 7.20.

### PACKAGING:

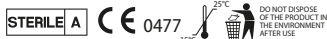
Bottle of 10 ml, preservative free in Novelia® System.  
Bottle of 5 ml, preservative free in Novelia® System.  
Bottle of 3 ml (free sample), preservative free in Novelia® System.

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#### FABRICANT: VISUfarma SpA
#### DISTRIBUTEUR: VISUfarma B.V. - Amstelplein 1
#### SITE DE PRODUCTION: Omisan Farmaceutici S.r.l.