Relief Insert[®] 2.0

EN Foot-Positioning Orthosis
DE Fußlagerungsorthese
ES Ortesis de posicionamiento del pie
FR Orthèse de positionnement du pied
IT Ortesi di posizionamento del piede
NL Voetimmobilisatie-orthese

DA Fodlejringsortose
SV Fotavlastningsortos
PL Orteza pozycjonująca stopy
SL Ortoza za imobilizacijo stopala
HR Ortoza za imobilizaciju stopala
PT Ortese de posicionamento do pé

DARCO

**UK
CA**

- EN Certification mark declaring this product's compliance with UK legislation
 DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der Gesetzgebung des Vereinigten Königreiches bestätigt
 ES Marca de certificación que declara la conformidad de este producto con la legislación de GB
 FR Marque de certification attestant de la conformité de ce produit à la législation de l'RU
 IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione del Regno Unito
 NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de VK
 DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med UK-lovgivningen
 SV Märket deklarerar att produkten uppfyller lagarna i Storbritannien
 PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami Wielkiej Brytanii
 SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo Združenega kraljestva
 HR Certifikacijska oznaka koja označava sukladnost ovog proizvoda sa zakonodavstvom UK
 PT Marca de certificação que declara a conformidade deste produto com a legislação do Reino Unido

CE

- EN Certification mark declaring this product's compliance with EU legislation
 DE Zertifizierungszeichen, das die Übereinstimmung dieses Produktes mit der EU-Gesetzgebung bestätigt
 ES Marca de certificación que declara la conformidad de este producto con la legislación de la UE
 FR Marque de certification attestant de la conformité de ce produit à la législation de l'UE
 IT Marchio di certificazione che dichiara la conformità del prodotto con la legislazione UE
 NL Certificeringskenmerk dat aangeeft dat dit product voldoet aan de wetgeving van de EU
 DA Certificeringsmærke, der erklærer dette produkts overensstemmelse med EU-lovgivningen
 SV Märket deklarerar att produkten uppfyller lagarna i EU
 PL Znak certyfikacyjny deklarujący zgodność tego produktu z przepisami UE
 SL Certifikacijska oznaka, ki potrjuje skladnost tega izdelka z zakonodajo EU
 HR Certifikacijska oznaka koja označava sukladnost ovog proizvoda sa zakonodavstvom EU
 PT Marca de certificação que declara a conformidade deste produto com a legislação da UE

A EN Strap loops
DE Umlenkgeschlaufen
ES Lazos de desviación
FR Boucles de sangles
IT Passanti
NL Doorhaallus
DA Stropper
SV Remögla
PL Klamry do przewlekania
SL Zanke
HR Skretne omče
PT Fivetas

B EN Forefoot strap
DE Vorfußgurt
ES Correa del antepié
FR Sangle d'avant-pied
IT Cinturino sull'avampiede
NL Voorvoetband
DA Forfødsrem
SV Framfotsrem
PL Pasek na przedniej części stopy
SL Trak za sprednji dio stopala
HR Pojas za prednji dio stopala
PT Correja de antepé

C EN Pump to fill the air liner air chambers
DE Pumpe zum Befüllen der Luftkammern des Air-Liners
ES Bomba para inflar las cámaras de aire del forro inflable
FR Pompe de gonflage des alvéoles de l'Air-Liner
IT Pompa per il riempimento delle camere d'aria dell'Air-Liner
NL Pomp voor het vullen van de luchtkamers van de air-lijnen
DA Pumpe til at fylde luftkammerne i Air-linieren
SV Pump för påfyllning av luftkammrarna i Air-Liner
PL Pompa do napełniania komór powietrznych wkładki powietrznej
SL Črpalka za polnjenje zračnih komor na vložku Air-Liner
HR Pumpa za punjenje zračnih komora zračne podstave
PT Bomba para encher as câmaras de ar do revestimento Air

D EN Valve for releasing the air from the air chambers
DE Ventil zum Ablassen der Luft aus den Luftkammern
ES Válvula para desinflar las cámaras de aire
FR Valve d'évacuation de l'air des alvéoles
IT Valvola per lo scarico dell'aria dalle camere d'aria
NL Ventiel waardoor lucht uit de luchtkamers kan ontsnappen
DA Ventil til udlædning af luft fra luftkammerne
SV Ventil för att släppa ut luft ur luftkammarna
PL Zaworek do spuszczenia powietrza z klamery powietrznych
SL Ventil za izpuščanje zraka iz zračnih komor
HR Ventil za ispuštanje zraka iz zračnih komora
PT Válvula de saída do ar das câmaras de ar

E EN Calf splint with integrated air liner with air chambers
DE Waden-Schiene mit fest integrierten Air-Liner mit Luftkammern
ES Férula para pantorrilla con forro inflable integrado fijo y cámaras de aire
FR Attelle de mollet avec Air-Liner intégré et alvéoles
IT Stecca per polpaccio con Air-Liner integrato fisso e dotato di camere d'aria
NL Kuibrace met vast geïntegreerde air-lijnen met luchtkamers
DA Læg-skinne med fast integreret Air-Liner med luftkammer
SV Vadskinn med fast Air-Liner med luftkammare
PL Szyna łydkowa z umocowaną na stałe wkładką powietrzną z komorami powietrznymi
SL Opornica za meča s fiksno integriranim vložkom Air-Liner z zračnimi komorami
HR Udlaga za list s čvrsto ugrađenom zračnom podstavom sa zračnim komorama
PT Tala dos gêmeos com revestimento Air integrado de forma fixa com câmaras de ar

F EN Automatic modification of the width: For unhooking, if necessary
DE Automatische Weitenregulierung: Im Bedarfsfall zum Aushaken
ES Regulación automática de la anchura: Se puede desenganchar en caso necesario
FR Réglage automatique de la largeur : Si nécessaire pour décrocher
IT Regolazione automatica della larghezza: Se necessario per sganciare
NL Automatische wijdeaanpassing: Waar nodig los te haken
DA Automatisk bredde regulering: Kan hæftes af efter behov
SV Automatisk vidreglering: Kan hakas loss vid behov
PL Automatyczna regulacja szerokości: W razie potrzeby można rozpiąć
SL Samodejna regulacija širine: Za odpenjanje po potrebi
HR Automatska regulacija širine: Može se otkopčati po potrebi
PT Regulação automática em largura: Se necessário, para desenganchar

G EN Calf liner
DE Waden-Liner
ES Forro para pantorrilla
FR Doublure du mollet
IT Rivestimento del polpaccio
NL Kuivvoering
DA Læg-lijner
SV Vadlinier
PL Wkładka na łydce
SL Vložek za meča
HR Podstava za list
PT Revestimento dos gêmeos

H EN Air liner with air chambers
DE Air-Liner mit Luftkammern
ES Forro inflable con cámaras de aire
FR Air-Liner avec alvéoles
IT Air-Liner con camere d'aria
NL Air-Liner met luchtkamers
DA Air-Liner med luftkammer
SV Air-Liner med luftkammare
PL Wkładka powietrzna z komorami
SL Air-Liner z zračnimi komorami
HR Zračna podstava sa zračnim komorama
PT Revestimento Air com câmaras de ar

UK REP

CH REP

EN Responsible Person in the UK
DE Verantwortliche Person im Vereinigten Königreich
ES Persona responsable en el Reino Unido
FR Personne responsable Royaume-Uni
IT Rappresentante Autorizzato nel Regno Unito
NL Verantwoordelijke rechtspersoon VK
DA Befuldsmægtiget i Det Forenede Kongerige
SV Auktoriserad representant i Storbritannien
PL Osoba odpowiedzialna w Wielkiej Brytanii
SL Odgovorna oseba v Združenem kraljestvu
HR Odgovorna osoba u UK
PT Responsável no Reino Unido

EN Authorised Swiss representative
DE Bevollmächtigter in der Schweiz
ES Representante autorizado en Suiza
FR Mandataire en Suisse
IT Rappresentante Autorizzato Svizzero
NL Gefulmachtigde in Zwitserland
DA Befuldsmægtiget i Schweiz
SV Anmält organ i Schweiz
PL Autoryzowany przedstawiciel w Szwajcarii
SL Pooblaščen zastopnik v Švici
HR Opuномоћnik u Švicarskoj
PT Representante autorizado na Suíça

EN Store in a dry place
DE Trocken aufbewahren
ES Conservar en un lugar seco
FR Stocker au sec
IT Conservare in luogo asciutto
NL Droog bewaren
DA Skal opbevares tørt
SV Förvaras torr
PL Przechowywać w suchym miejscu
SL Shranjajte v suhem prostoru
HR Čuvati na suhom mjestu
PT Conservar seco

I EN Front splint with front liner
DE Front-Schiene mit Front-Liner
ES Férula frontal con forro frontal
FR Attelle avant avec doublure avant
IT Stecca anteriore con rivestimento anteriore
NL Frontbrace met frontvoering
DA Front-skinne med Front-lijner
SV Frontskenna med frontliner
PL Szyna frontowa z wkładką frontową
SL Sprejdnja opornica s sprednjim vložkom
HR Prednja udlaga s prednjom podstavom
PT Tala dianteira com revestimento dianteiro

Relief Insert® 2.0 | Foot-Positioning Orthosis**Important notes**

If there is a serious deterioration in your state of health as a result of using the medical device, please report this to your specialist dealer or the manufacturer as soon as possible. If you report this to the specialist dealer, the dealer will forward this immediately. The reports are agreed with and processed by the competent authorities.

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. You must not drive a vehicle while you are wearing this product.

This aid is allowed for single-patient use only. It does not need to be cleaned or reprocessed between uses. There is a risk of thrombosis due to the restricted ankle movement.

The air chambers must not be over inflated to prevent pressure points on the leg and to prevent the air chambers from bursting. The air liner with pump cannot be washed.

Design features

The lower leg / foot orthosis is available in 3 sizes with a universal fit for right and left.

The splinting system is used to position the lower leg / foot. The exchangeable insole enables patients to individually adjust the fit. The orthosis is fixed to the leg with three circular hook and loop fasteners.

The positioning orthosis can be put together with the Relief Dual® Plus off-loading shoe to form a walker.

Maintenance

The product does not require maintenance.

Indications

For general use in conservative and postoperative therapy of lower leg, ankle and foot injuries:

- > Pre- and postoperative support
- > Pes equinus prophylaxis
- > Post-traumatic stabilisation / Immobilisation
- > Immobilisation following an accident
- > Short-term stabilisation (ligament rupture, luxation, fractures, wounds, strains, sprains, etc.)

Contraindications without heel wedges

- > Strong plantar flexion elasticity
- > Plantar flexion contracture of more than 10°
- > Shortening of the calf muscles

Purpose

Orthosis for positioning and/or fixing the foot in a defined position, which serves to immobilise and securely position the affected body parts and joints.

Combination with other products

- > PegAssist® Insole
- > Puzzle Insole®
- > PegContour® Insole Uni
- > Relief Contour Insole
- > DAW (DARCO Achilles Wedges)
- > Relief Insert® 2.0 Lining
- > Relief Dual® Plus

Technical data**Sizes**

S	37.0 – 40.5
M	41.0 – 43.0
L	43.5 – 47.0

Colour

Grey-Blue

Material

Insole grey: EVA
 Lining and forefoot strap: Polyester
 Hook and loop straps: Nylon
 Adhesive: Polyurethane glue
 Splints: Polyamide / TPU
 Pump: TPU

To avoid the hook and loop straps getting in the way during the individual application steps, attach them behind onto the respective fleece side.

Using the Relief Insert® 2.0 as a splinting system:

1. Fold the front splint away to the side.
2. If desired, another compatible DARCO insole can be placed in the splinting system. To do this, remove the insole. Then place and fix the desired insole analogously to the insole in the splinting system.
3. Place the leg on the calf liner of the calf splint. Make sure that the foot is in contact with the insole.
4. The width of the guide splint automatically adjusts to the patient's needs by means of the adjuster on the back of the guide splint.
5. Once the leg is positioned, close the splinting system. To do this, place the front splint on the shin. Make sure that the front splint is positioned centrally within the calf splint. Guide the three hook and loop straps through the loops. First close the lowest strap at the ankle and then the two upper hook and loop straps. Pull the hook and loop straps as tight as necessary. Ensure that the leg is supported comfortably. Optionally, the blue forefoot strap (B) can be attached to the forefoot for additional support.
6. To achieve additional stabilisation of the ankle joint, the volume of the air chambers can be increased using the pump attached to the side. To reduce the volume, release the air using the silver valve. This allows for individual filling and adjustment. Ensure that the leg is supported comfortably. If impaired sensation, swelling or pain occurs, the pressure should be reduced by releasing air from the chambers using the valve. Should persistent pain occur, the treatment should be interrupted and a doctor should be consulted.

Changing the lining

Removing the lining:

7. Remove the blue calf liner attached to the calf splint with hook and loops.
8. The grey air liner with integrated air chambers and pump remains in the product and must not be removed.
9. Now carefully remove the blue front liner attached to the front splint with hook and loop fasteners.
10. The calf liner, front liner and forefoot strap can be washed, if required.

Fitting the lining:

After cleaning the blue lining, please carry out the steps described below:

11. First attach the front liner to the front splint using the hook and loop fasteners. To do this, fold the liner in the middle lengthwise and place it centrally (according to the shape of the front splint) on the front splint. By unfolding the liner, it clings to the front splint in the correct position. Gently press the front liner against the splint.

12. The calf liner is also folded lengthwise in the middle for attachment to the splint. Place the calf liner in the centre of the calf splint with the notches at the level of the ankle. By unfolding the liner, it clings to the calf splint in the correct position. Gently press the calf liner.

To close the splinting system or walker, follow the steps described from step 1.

Important notes

The Relief Insert® Splinting System may only be used in combination with the Relief Dual® Plus Off-loading Shoe for walking. The splint is not designed for walking without a shoe. It may only be used for positioning.

Care instructions

Washing

Splint, insole: If these are slightly soiled, wipe with a damp cloth or clean with commercially available disinfectants.

Liners: To wash, the blue calf and front liners are removed from the splint and then washed at approx. 60° C in a washing machine. The optional forefoot strap can also be washed at 60° C. The air liner with air chambers and the pump cannot be washed.

Drying

Do not dry any of the components of the Relief Insert® 2.0 in a dryer or on a radiator, but dry them at room temperature.

Working life / Lifespan of the product

The lifespan of the medical device is determined by natural wear and tear due to proper handling / cleaning. The medical device is designed to be used for the usual duration of use prescribed by a doctor without showing a reduction in safety or performance. After the period of use prescribed by your doctor, check the function of the product together with your doctor or specialist dealer.

Disposal

Dispose of after use. Please dispose of the packaging in an environmentally friendly manner. Dispose of this product after use in accordance with applicable local laws and regulations.

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V-M Orthotics Ltd.
ALBO-Healthcare GmbH