## ManukaLind Salbe 🇬🇧

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## ManukaLind Ointment 🇬🇧

**Wound ointment. To be applied to the skin. Made with Manuka honey.**

ManukaLind is a medical product for wound care and can be used at all stages of wound healing.

- Dressing changes are less painful and less traumatizing as the ointment reduces dressing adhesion due to ongoing moisture.

- ManukaLind creates an ideal moist and optimum pH wound environment.

- ManukaLind has antibacterial properties due to its high osmolarity and protects the wound from invading germs due to its high viscosity.

- ManukaLind promotes autolytic wound debridement to cleanse the wound and remove scabs and dirt.

- ManukaLind counteracts wound odor and promotes wound granulation.

### Instructions for use

- Can be applied as needed several times a day for wound care.

- Gently clean the edges of the wound where needed.

- Apply ManukaLind liberally. There must be full contact between ManukaLind and the wound bed. Apply ManukaLind directly into wound cavities.

- A suitable sterile dressing may be used to cover the treated wound.

- If interactive dressings are used (e.g., polyfoam or hydrocolloids), care must be taken to ensure that they do not contain active substances (e.g., hydrogel dressings may contain specific active substances) because there is no available experience as regards compatibility with ManukaLind.

- The dressing should be changed if it is saturated with exudate, or if the covering dressing leaks wound secretions.

- When removing/changing the dressing, if necessary, rinse/clean the wound with water and reapply ManukaLind.

- The currently valid guidelines for hygiene and infection prevention must be observed when caring for wounds or changing dressings. The minimum that these include are thorough hand disinfection, the use of sterile disposable gloves and mouth/ nose protection.

### Areas of application / Indications

ManukaLind can be used for skin irritation and wounds. These include: Acute and chronic wounds and ulcers, infected wounds, surgical wounds, operation wounds, burns (1st and 2nd degree) and scalds, necrotic wounds, diabetic wounds, covered wounds, decubitus ulcers, poorly healing wounds, cut wounds and abrasions, superficial wounds, scar care on sutured wounds, skin irritation, foot and leg ulcers (venous, arterial).

### Contraindications

- Not to be used in patients in the instance of a known allergy to bee products (e.g. honey, pollen, propolis etc.), thyme or lavender oil.

- Do not use on heavily weeping wounds, third degree burns, in deep tight cavities/wound areas or sinuses.

- Not suitable for children under 12 months and pregnant or nursing women.

- There are no reports of elevated blood sugar levels in patients with diabetes after use of the ointment, as the product is not reabsorbed. Nevertheless, diabetics should consult with their doctor before using ManukaLind.

### Safety Precautions and Warnings

- Do not bring the opening of the tube into direct contact with the wound. Only use one tube per patient.

- In case of contact with eyes, rinse thoroughly.

- After application, light burning or fluid withdrawal may occur. In case of persistent complaints remove dressing, rinse the wound with water and consult your doctor if necessary.

- If allergic reactions occur, stop treatment immediately and contact your doctor.

- If the desired effect is not achieved within 3 weeks, discontinue use and consult your doctor.

### Ingredients

Macrogol, Manuka honey, essential oils (lavender oil, thyme oil).

### Disposal

Empty packaging can be disposed of with household waste.

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