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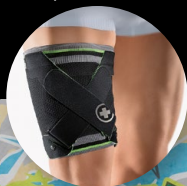
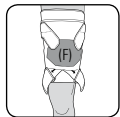
The **MyoActive Sport from BORT** is a high-quality thigh support. It combines the operating principle of compression, conventional tape and Kinesio tape in a single support. The thigh support can be worn as a preventive measure when the muscles are under heavy strain. It ensures better circulation for the muscles and therefore an improvement of movement and stamina. During the acute phase, both compression and the tape principle ensure pain reduction and stabilisation of the weakened or injured thigh structures.

The **BORT MyoActive Sport** can also be used in case of chronic pain. Here, the operating principles of compression, conventional tape and Kinesio tape integrated in the support relieve muscle structures which are already damaged and provide protection against new muscle lesions.

COOLMAX® fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

Instructions for use:

A circular belt system which you can attach to the thigh support (step 6) is provided with it.

1. Open both cross restraints and let the straps hang downwards. Here, you see the green velour inner surface with the directional arrows and marking strips as a so-called "pull stop".
2. Put the upper thigh support on when standing and bend your knee slightly. Here, the function zone (F) should be at the same height as the injured region.

Tip: The thigh support can be used for the knee-stretching (front) and knee-bending (rear) muscles.

3. Then pull the first cross restraint downwards to the marking strip (see arrow). Here, the wide, red marking strip indicates the maximum pull (which must not be exceeded).

4. Then attach the cross restraint to the upper velour diagonally using Velcro.

5. Repeat the process with the second cross restraint. Here, the straps cross as an X. It is not important which cross restraint you attach first.

6. If required, the included circular belt fasteners can be attached to the velour lugs with Velcro under tensile stress. Ensure that you attach the longer strap above and the shorter strap below.

Indications:

Preventive: For highly strained muscle groups.

Acute: Swollen contusions with haematoma and muscle injuries.

Chronic: Protection against new muscle lesions, reduction of the burden on damaged muscle structures (e.g. scarring after old injuries), following muscle injuries.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body, latex allergy.

Composition:

38% elastodiene, 34% polyester, 19% polyamide, 9% elastane

Important Note:

No re-use – this medical aid is intended for treating one patient.

If the thigh support is to be worn at night, observe that circulation is not impeded. In case of feelings of numbness, loosen the thigh support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

The product contains latex and may trigger allergic reactions

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Fix the Velcro closure to avoid damage to other laundry. Pull into shape and air-dry. Avoid heat.

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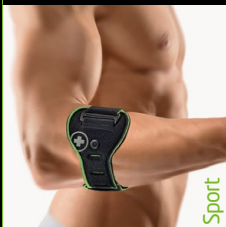
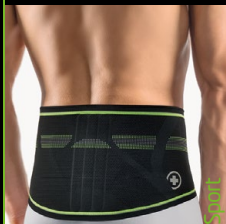
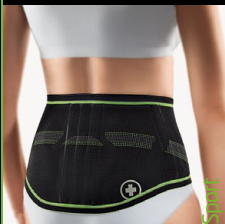
EN The **BORT MyoActive Sport thigh support** provides support and stabilises the musculature with its integrated operating principle consisting of compression, conventional tape and Kinesio tape. The high-tech **COOLMAX®** fibre wicks moisture away from the body for dryness and is comfortable to wear. Innovative materials, processed using state-of-the-art manufacturing technology, provide the BORT Sport supports ideal properties for use in sports and therapeutic application.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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