
en

## INSTRUCTIONS FOR USE TULIPS NIPPLE SHIELDS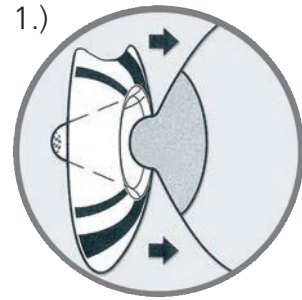

### Package contains:

Two silicone nipple shields, size M (20 mm) or L (24 mm), one storage box, one copy of the instructions for use.

### Indication:

Tulips nipple shields are suitable for short-term use to help with latching on and to protect sore, sensitive nipples during breastfeeding.

### Use:

- Moisten the inside of the nipple shield with a little breast milk or water, or use a lanolin nipple cream, so that it adheres more easily to the breast. Pull back the edge of the nipple shield and centre the shield over the nipple. The baby's nose should be at the tulip-shaped cut-out, so that it is touching the mother's skin (image 1).
- Release the edges of the nipple shield onto the breast (image 2).

### Notes:

Nur only when breastfeeding, remove after use and clean according to the instructions. Frequent sterilisation and boiling causes the material to wear faster. It is recommended to let the nipple shields dry before storing them in the storage box. Stop using the nipple shields if they show any signs of damage. Store the nipple shields and accessories out of the reach of children. Only wear the nipple shields temporarily while breastfeeding, and clean after use according to the instructions. If you have any questions, please contact a healthcare specialist. Tulips nipple shields are latex free.


### Cleaning/sterilisation:

At home: Boil the nipple shields in water for ten minutes before the first use, and then once per day thereafter. Clean with washing-up liquid after each use, then rinse well and allow to dry on a clean cloth. The nipple shields and storage box are dishwasher safe.

In a hospital environment: Nipple shields: sterilisation 121°C = 20 min. / 134°C = 18 min. and thermal disinfection 93°C = 10 min. Storage box: thermal disinfection 93°C = 10 min.

### Disposal:

harmless, in accordance with local regulations.

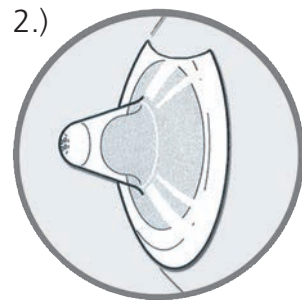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
INSTRUCTIONS FOR USE <p>TULIPS NIPPLE SHIELDS</p>

<b>Indications:</b> <p>Tulips nipple shields are suitable for short-term use to solve breastfeeding problems such as sucking problems and nipple injury, sensitivity, etc.</p>
<b>Use:</b> <ol style="list-style-type: none"><li>To ensure the nipple shield sticks better to the breast, you can use a little breast milk or water to moisten the contact surface of the nipple shield, or use lanolin nipple cream. Pull the edge of the nipple shield back; make sure the center is aligned with the nipple. Make sure the baby's nose can breathe smoothly and can touch the mother's skin (Figure 1).</li> <li>Turn the edge of the shield back towards the breast (Figure 2).</li></ol>
<b>Notes:</b> <p>Only use during breastfeeding. After use, please clean according to the instructions. Frequent sterilization and boiling will accelerate material aging. It is recommended to store the nipple shield before storing it in the storage box, and to dry it thoroughly. If there are signs of damage to the nipple shield, please stop using it. Please store the nipple shield and its accessories in a place away from children. Only use the nipple shield for a short time during breastfeeding. After use, please clean according to the instructions. If you have any questions, please contact a professional.</p>
<b>Cleaning / Sterilization:</b> <p>At home: Before the first use, boil the nipple shield in water for 10 minutes, and then boil once a day for 10 minutes. After each use, please use detergent to clean, and then wash thoroughly with water and place on a clean cloth to dry. The nipple shield and storage box can be washed in the dishwasher.</p>
<b>In the hospital:</b> <p>Nipple shield: Sterilization 121°C = 20 min / 134°C = 18 min, and high temperature disinfection 93°C = 10 min. Storage box: High temperature disinfection 93°C = 10 min.</p>
<b>Disposal:</b> <p>harmless, in accordance with local regulations.</p>

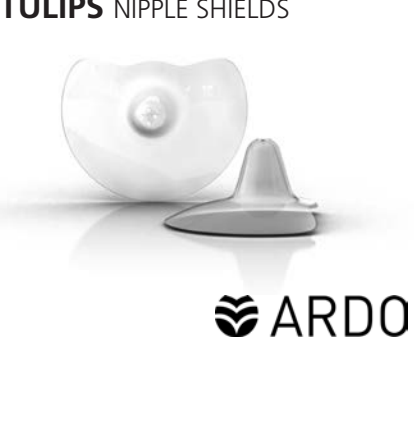ARDO

Ardo medical AG Gewerbestrasse 19 6314 Unterägeri Switzerland T +41(0)41 754 70 70 F +41(0)41 754 70 71 info@ardo.ch www.ardomedical.com
 0123 Class 2a

Art.No. 99.03.95Z 10/2013