Original

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- Moxarolle Moxazigarre
- Moxarol Moxacigar

Artikel Nr.

MED1000446
EAN 4250201204523

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- Moxarolle raucharm MED1000880
- Moxarol smokeless
- Moxa Kegel MED1000881
- Moxa Cone
- Moxa Löscher MED1000754
- Moxa Extinguisher EAN 4250201204530

Inhaltsstoffe · INCI

Beifußkraut
Folium Artemisia Argy

Ingredients · INCI

Mugwort leaves
Folium Artemisia Argy

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The moxa rolls must only be used by persons with sufficient medical training who can guarantee proper and effective treatment. The moxa rolls are to be used once for the manual warming of acupuncture points.

Instruction

After igniting the moxa roll by hand or by using a moxa holder, carefully bring the roll into the vicinity of the acupuncture point to be warmed and carry out the warming procedure. If the heat is too great, increase the distance between the ember and the skin. The best effect is achieved with a uniformly glowing ember. Remove the ash from the tip of the moxa roll at regular intervals. If this becomes unpleasant, leave the acupuncture point. After use, extinguish the moxa roll in a moxa extinguishant, sand or water. As a precaution, always provide a fireproof container. 20–30 minutes is sufficient for a normal treatment. After the treatment, you should rest and keep the treated point warm.

Warnings

- Moxa gets very hot and may cause burns.
- Carefully place ash in a fireproof container
- Do not use moxa rolls, which are defective or broken
- Do not touch the skin with the burning moxa roll.
- Do not fall asleep during use
- Do not take narcotics, anaesthetics or alcohol before or during the treatment
- Do not move during the treatment
- Be aware of the risk of burns from the moxa ash.
- Extinguish the moxa immediately after use
- Do not use near the face
- Be careful when using near hair
- Keep out of the reach of children
- For external use only