Norditalia

IT	MANUALE D'USO	3
EN	OPERATING MANUAL	6
PL	INSTRUKCJA OBSŁUGI	9
EL	ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ ΧΡΗΣΗΣ ΓΙΑ	12
TR	BELİRTİLEN İÇİN KULLANIM KILAVUZLARI	15
RO	MANUAL DE UTILIZARE	18
FR	MANUEL D'UTILISATION	21
DE	GEBRAUCHSANWEISUNG SHEFT	24
HU	HASZNÁLATI ÚTMUTATÓ	27
NL	HANDLEIDING VOOR DE GEBRUIKER	30
RU	ИНСТРУКЦИЯ	33
ES	MANUAL DE USO	36
PT	MANUAL DE INSTRUÇÕES	39
SR	UPUTSTVO ZA UPOTREBU	42
UK	ІНСТРУКЦІЯ	45
BG	РЪКОВОДСТВО ЗА УПОТРЕБА	48
CZ	NÁVOD K POUŽITÍ	52
SK	NÁVOD NA POUŽITIE	55
VI	HƯỚNG DẪN SỬ DỤNG	58
AR	دليل التشغيل	64

Ariane P1 - Ariane Power - Ariane Plus
Ariette - Drop - Hi-Neb - @Neb
Meganeb - Meganeb Plus

CONSERVARE CON CURA QUESTO MANUALE
 KEEP THIS MANUAL IN A SAFE PLACE
 NINIEJSZĄ INSTRUKCJĘ OBSŁUGI NALEŻY STARANNIE PRZECHOWYWA
 ΦΥΛΑΞΤΕ ΠΡΟΣΕΚΤΙΚΑ ΑΥΤΟ ΤΟ ΕΓΧΕΙΡΙΔΙΟ
 BU KILAVUZU ÖZENLE SAKLAYIN
 PASTRATI ACEST MANUAL INTR-UN LOC SIGUR
 CONSERVER CE MANUEL AVEC SOIN
 DIESES HEFT SORGFÄLTIG AUFBEWAHREN
 TARTSA BIZTONSÁGOS HELYEN A HASZNÁLATI ÚTMUTATÓ
 BEWAAR DEZE HANDLEIDING ZORGVULDIG
 БЕРЕЖНО ХРАНИТЕ ЭТУ ИНСТРУКЦИЮ
 CONSERVAR ESTE MANUAL CON CUIDAD
 CONSERVAR ESTE MANUAL COM ATENÇÃO
 SACUVAJTE OVO UPUTSTVO
 ДБАЙЛИВО ЗБЕРІГАЙТЕ ЦЮ ІНСТРУКЦІЮ
 СЪХРАНЯВАЙТЕ ТОВА РЪКОВОДСТВО НА БЕЗОПАСНО МЯСТО
 UCHOVÁVEJTE TENTO NÁVOD NA BEZPEČNÉM MÍSTĚ
 UCHOVÁVAJTE TENTO NÁVOD NA BEZPEČNOM MIESTE
 LƯU GIỮ, BẢO QUẢN HƯỚNG DẪN CẨN THẬN ĐỂ SỬ DỤNG LÂU DÀI.
 احفظ بهذا الدليل في مكان آمن

Norditalia Group S.r.l.

Via Colli Storici, 215

25015 Desenzano del Garda (BS) - ITALY

Tel.: +39 030 99 100 20 - Fax: +39 030 99 101 47

Web: www.norditalia.biz - Mail: info@norditalia.biz

NUMERO VERDE
800 642 609

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OPERATING MANUAL

EN

AEROSOL THERAPY SYSTEM FOR DOMESTIC USE

INTENDED USE

The aerosoltherapy device is a medical device used in combination with medicines and destined to be used on human beings for a therapy or lessening of a respiratory disease. There are currently no contraindications regarding the use of the medical device. The main side effects may be caused by the medicine used.

PATIENTS AND USERS

The device is intended for patients suffering from respiratory problems to which a doctor has prescribed the use. The user could be the patient himself or, if the patient isn't able to perform the therapy independently, the one who assists the patient. The patient and the user must fully understand the contents of the instruction manual and be able to carry out the necessary operations for the correct use of the device following the doctor's prescription.

PRINCIPLE OF OPERATION OF THE APPLIANCE

The device consists of a system composed of a nebulizing ampoule connected to a compressor that uses the air flow to nebulise the medicament in very fine particles that are taken by the patient during the breath by using appropriate accessories.

ESSENTIAL PERFORMANCE: the "Amount nebulized" with open plug (3) higher than or equal to 50% of declared value.

IMPORTANT WARNINGS: Failure to follow the warnings and the content of the present manual can expose to heavy risks such as: healthstate worsening, scald, fire, electrocution, choking, cross-infections, re-infection, death, environmental contamination. Failure to comply with the warnings and with the whole contents of this manual may result in a decrease of the performance of the device, in failure to function and to compromise safety.

- Read all the instructions carefully before using the appliance;
- Respect the electrical specifications indicated on the appliance; If necessary, use adapters or extension leads that are compliant with the safety standards in force, paying attention not to exceed the rated current value and the maximum power value indicated on the adapter;
- The inhalation of any substance must be prescribed and monitored by the doctor, who will indicate the most suitable doses and treatment methods. The therapeutic effectiveness of the treatment must always be verified by the doctor;
- Electricity supply failure, an unexpected fault or any other unfavourable condition may cause the appliance not to work; we therefore recommend that you have an appliance or medicine (subject to agreement with your doctor) that may be used as an alternative;
- Only use the appliance for the use foreseen (aerosol inhalations) and as described in this manual;
- Do not let children or incapable persons use the appliance without supervision; in such cases, pay special attention during treatment;
- Do not use the appliance if the appliance housing or power supply cable are damaged;
- Never leave the appliance and all its parts, including packaging, within the reach of children, incapable people, domestic animals or insects.
- The elastic bands (optional) can include latex (LATEX). Latex can produce allergic reactions.
- Do not use the appliance with wet or damp hands or while taking a bath; Do not wet the appliance with water or other liquids; do not use the appliance if it has accidentally become wet;
- Keep the appliance and the power supply cable away from sources of heat; The appliance is not suitable for use in the presence of inflammable substances;
- Do not use the appliance in rooms where sprays have just been used; air the room well before proceeding with the therapy;
- Do not block or obstruct the appliance air openings and do not place it on a soft surface such as a bed or settee;
- Do not insert any objects into the cooling openings; Do not use the appliance in cases where it makes unusual noises;
- To guarantee the performances, functionalities and safety of the appliance, only use Norditalia original accessories and spare parts.
- We recommend personal use of the accessories; In order to reduce the risk of infection, we recommend to disinfect the accessories before each use.
- Do not pull the power supply cable or the appliance to remove the plug from the power supply socket. Always remove the plug after use. In case of detachable cable always remove the plug from the wall socket before detaching the cable from the appliance.

LATEX

- Do not leave the appliance exposed to atmospheric agents; store it in the environmental conditions indicated.
- Do not wrap the tube or the power cable around your neck.
- The appliance include some small parts which can be dangerous if inhaled or swallowed.
- Do not operate the appliance in combination with other appliances and accessories not described in the present manual.
- Do not operate the device if stored in dirty places.
- Do not use the device in different environmental conditions than those provided for.
- Before using the device, keep it in the environmental conditions provided for at least for 24 hours.
- It is forbidden to tamper the device or any of its parts.
- Any repairs or maintenance must be carried out exclusively by the manufacturer or by their authorized personnel.

RESPONSIBILITY

The manufacturer, his representative, or the importer shall only be considered responsible in terms of safety, performance and reliability if: - the assembly technique, calibration, repairs or amendments have been performed by persons authorised by them; - the electrical system complies with the standards in force; - the operating instructions have been respected; The manufacturer declines all responsibility for damage deriving from improper, erroneous and unreasonable use. By the time the product may deteriorate. After 5 years have passed from the manufacturing date stated on the data plate label the product is no more usable and the manufacturer cannot be held responsible for damages or incidents caused by the product.

ELECTROMAGNETIC COMPATIBILITY

The appliance is compliant with the electromagnetic compatibility standards currently in force and is suitable for use in all buildings, including those for residential / domestic use. The RF emissions from the appliance are very low and are not likely to cause any interference with nearby appliances.

In any case, we recommend that you avoid positioning it on top of or near other appliances. If it does appear to be interfering with your electrical appliances, move it or connect it to a different electrical socket. Radio appliances may affect the functioning of the appliance, keep them at least 3 m. away.

For further information please check our website www.norditalia.biz.

OPERATING INSTRUCTIONS

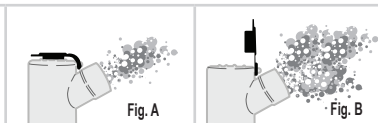
1. Position the appliance on a stable surface.
2. When using the appliance for the first time, clean the accessories as indicated in the "cleaning the accessories" section.
3. Connect the tube (1) to the appliance and make it work for 1 minute to release any humidity. Connect the nebulizing ampoule (2) to the appliance, using the tube (1).
4. Connect one of the desired accessories to the nebulizing ampoule: mouthpiece (4), nosepiece (5) or mask (6), as indicated in the accessory connection diagram on page 2.
5. Insert the medication into the nebulizing ampoule according to the filling limits indicated in the accessories connection diagram.
6. Insert the plug into the power supply socket after having checked that the power supply voltage corresponds to the rated voltage indicated on the label located under the appliance.
7.  Start the appliance using the switch, bearing in mind that it may be tested for intermittent operation; The operating times are indicated in the table on page 3. The emission of vapour from the ampoule confirms correct functioning. When the appliance is new it is normal for the compressor and accessories to have a peculiar smell which leaves after few uses.
8. Inhale the nebulized medicine until exhaustion. It is normal for a certain amount of medicine (approximately 0.5 ml.) not to be nebulized. The not nebulizing quantity, such as the nebulization speed, depend on the medicine characteristics. For example, drugs highly viscous or producing foam may be more difficult to nebulize.
9. Once the application has been completed, switch the appliance off and remove the plug from the power supply.
10. To avoid the risk of infection, clean the appliance and the accessories as indicated below.
11. In case of failure to function check the correct connection of plug to the power supply socket and that switch is in ON position. In case of failure to release the nebulization check the correct connection of accessories and the presence of all parts. The presence of dirt or limescale in the ampul (2) may cause failure to function. Clean it as indicated in the present manual.

Important instructions about using the nebulizing ampoule.

The plastic Neboplus ampoule (2) can be used in two different ways:

- with the plug (3) inserted, for slow nebulization (Fig. A);
- with the plug (3) open, for standard nebulization (Fig. B).

Attention: the nosepiece can only be used with the plug (3) inserted in the ampoule.

CLEANING THE APPLIANCE

All cleaning operations must be performed with the plug of the appliance removed from the power supply socket. To avoid anomalous appliance operation and the inhalation of undesired substances, clean the appliance and the accessories immediately after performing the treatment. The appliance and the accessory storage area must be cleaned using a cloth slightly moistened with denatured alcohol. After cleaning the appliance, wait until the alcohol completely evaporates before re-use.

Do not use other liquids or substances. Do not use excessively wet cloths, if liquids come into contact with the electrical parts of the appliance, operating defects or irreparable damage may be caused.

CLEANING THE ACCESSORIES

After use, and if necessary, before use, wash the accessories (not including the tube) carefully with tepid water and dry them with a cloth. Then put the accessories into a clean storage area. When cleaning, ensure that all residue is removed and that the accessories are dried properly. Under no circumstances should you use substances that may be toxic if they come into contact with the skin or mucous, if ingested or inhaled. Any residual humidity in the tube may be removed by leaving the aerosol on without the ampoule connected for a few minutes. If the tube is dirty it should be replaced. The accessories (not including the tube) may be disinfected in boiling water (100°C) for

at least 40' or in an autoclave (120°C) for at least 20'. Do not boil the tube. Do not put weights on the accessories and do not bend them; do not exceed the temperatures indicated;

If it is necessary to disinfect the accessories, adopt the methods approved by the regulations in force and in compliance with the resistance of the material (see Table 1: List of material). The accessories (not including the tube) may be disinfected using specific products such as the MILTON® method: they should be rinsed to remove all traces of disinfectant, dried and stored in a dry and clean environment. After disinfection, store clean bags.

PVC accessories will lose their transparency in water or vapour; this phenomenon will disappear after a few days in a dry environment and will not affect usability. All the accessories may be disinfected up to a maximum of 25 times, following which they should be replaced. The frequency and interval between one disinfection session and another should be established according to the discretion of the doctor and/or the clinic and according to the pathology. The accessories should normally be disinfected prior to each change of patient. Do not wash accessories in a washing machine.

In case of use onto high-risk groups (e.g. cystic fibrosis patients), consult your doctor who will prescribe specific precautions.

REPLACING THE FILTER

The appliance is fitted with a filter, which eliminates any impurities from the air sucked in by the compressor.

On a periodical basis, or if the appliance is no longer efficient, check the conditions of the filter, if it is very dirty it should be replaced. For checking or replacement, lift up the plug of the filter-holder compartment. If necessary, extract the filter using some tweezers. (See diagram on page 2).

EXPECTED LIFE

The expected life of the appliance and its accessories is as follows:

Device 5 years (one thousand hours of operation)

Ampul 12 months (every 6 months for frequent use - 3 applications per day)

Mouthpiece / Nasal / Mask / Tube 12 months

The life of the accessories vary according to the environment and the conditions of use. Frequent use of the product can shorten the expected life.

PERIODIC CHECKS

To preserve the efficiency of the appliance we recommend replacing the nebulizing ampoule every 12 months (every 6 months for frequent use - 3 applications per day). Damaged accessories should be replaced.

Although a correctly used appliance can last a very long time, we recommend to replace it after no more than 5 years or a thousand working hours from the manufacturing date. Periodically, or in case of doubt about the performances of the appliance, verify for the "Amount nebulized" with open plug (3) not to decrease below 50% of declared value. The test have to be performed using 4 ml of physiological solution. If the performances would be not compliant try replacing all the accessories and the filter or get another appliance.

WASTE DISPOSAL

Dispose of the appliance and its accessories in compliance with the regulation in force. Do not dispose of this appliance in the environment. In case of doubt please call your local authorities.

GUARANTEE CONDITIONS

This appliance is guaranteed for the period indicated in the table on page 3, except where otherwise indicated or for different local regulations.

The accessories (Table 1) are guaranteed for one year from date of purchase.

The guarantee is only valid if the device is returned to the stockist, together with its receipt as proof of the date of purchase.

The guarantee covers the appliance for any fabrication defects, on condition that the same has not been tampered with by the user or by persons other than the authorised maintenance service personnel. For service or for reporting abnormal operation please contact the manufacturer or your local distributor.

The guarantee does not include:

- faults or defects caused by improper use, or failure to comply with the operating instructions;
- interventions for presumed defects, checks on request, ordinary maintenance and components subject to natural wear and tear;
- accidental damage such as shocks, falls, etc.

NB. For repairs or overhauls, return the appliance to the stockist in clean condition.

KEY

- | | | | |
|-------------------------|----------------|-----------------------------|---|
| 1. Tube | 5. Nosepiece | 15. Bottom of nebuliser | 19. Tube connector |
| 2. "Neboplus" nebuliser | 6. Mask | 16. Pixper | 20. MULTISPEED (optional for MEGANEB MODEL) |
| 3. Plug for ampoule | 13. Filter | 17. Selector | |
| 4. Mouthpiece | 14. Filter cap | 18. Upper part of nebuliser | |

Environmental Conditions for use / storage	Temperature 5 ÷ 40 °C / -25 ÷ 70 °C	Relative humidity without condensation 15 ÷ 90% / < 90%	Atmospheric Pressure 700 ÷ 1060 hPa
--	--	--	--

 Class II appliance	 Type BF appliance	 Important warnings	 Consult the instruction manuali	 Alternated current	0 Off	I On	IP21 Input Protection of: - objects -> minimum size 12.5 mm. - Liquids -> Dripping water.	 Number of years covered by guarantee	 0051 EC mark	 Serial Number
 Noise	 Operating pressure	 Functioning times	 Amount nebulized (ml/min)	 Amount inhaled (ml/min) (A.O.R.E.N 13544-1)	 Suggestion	 Limits of temperature	 Humidity limitation	 Keep it dry	 Manufacturer	

Power supply voltage and frequency are indicated on the data plate label present in the device.

** Data variable on the basis of the medicine used. The measurements have been taken with the use of laser diffraction technology using a physiological solution. The data supplied may not be applicable to suspension or very viscose medicine, consult the drug manufacturer.

*** All the technical data in the operating manual are indicative.

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