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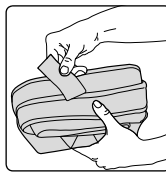
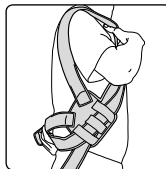
**Paso 1**

Saque la cinta de Velcro del envase.

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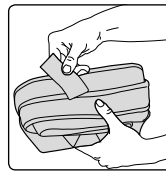
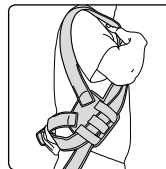
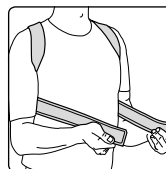
**EN****Instruction for use:****Step 1**

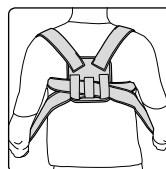
After unpacking the support, remove the white Velcro strap.

**Important:** You will need this Velcro strap to close the support after fitting to the body (step 6).**Step 2**

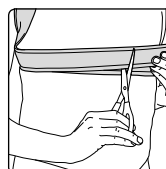
Put on the support like a backpack. Therefore slip with your arms through the loops. Be sure that the quadratic carrierplate is positioned on the back.

**Step 3**

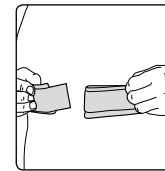
Put the crosswise running straps at the dorsum below the armpits from rear to front.

**Step 4**

Pull the straps tight around the shoulder.

**Step 5**

If required cut excess length of the frontally straps with a scissors.

**Step 6**

Close the straps at chest height with the earlier removed Velcro strap.

**Indications:**

Straightening and relief of the thoracic spine, i.e. misalignment of thoracic spine.

**Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

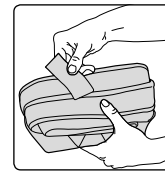
**Composition:**

55 % PES foam, 25 % polyamide, 20 % cotton

**Important Note:**

No reutilisation – this medical device is meant for the care of one patient only.

Please close Velcro fastening so that support fits tightly, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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