





MONTESOL[®]

Nasendusche
Nasal Douche
Douche Nasale
Doccia Nasale
Neusdouche
Назальный душ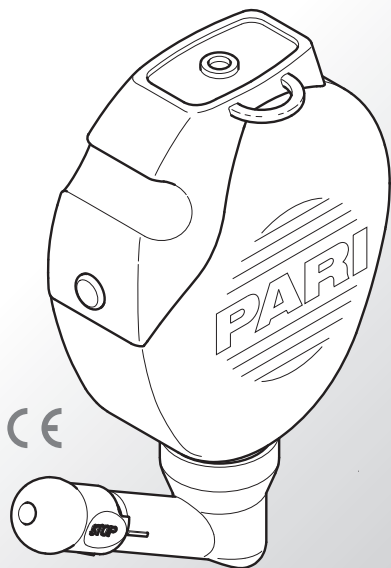

de	Gebrauchsanweisung.....	1
en	Instructions for use	9
fr	Notice d'utilisation	17
it	Istruzioni per l'uso.....	25
nl	Gebruiksaanwijzing	33
ru	Руководство по эксплуатации.....	41

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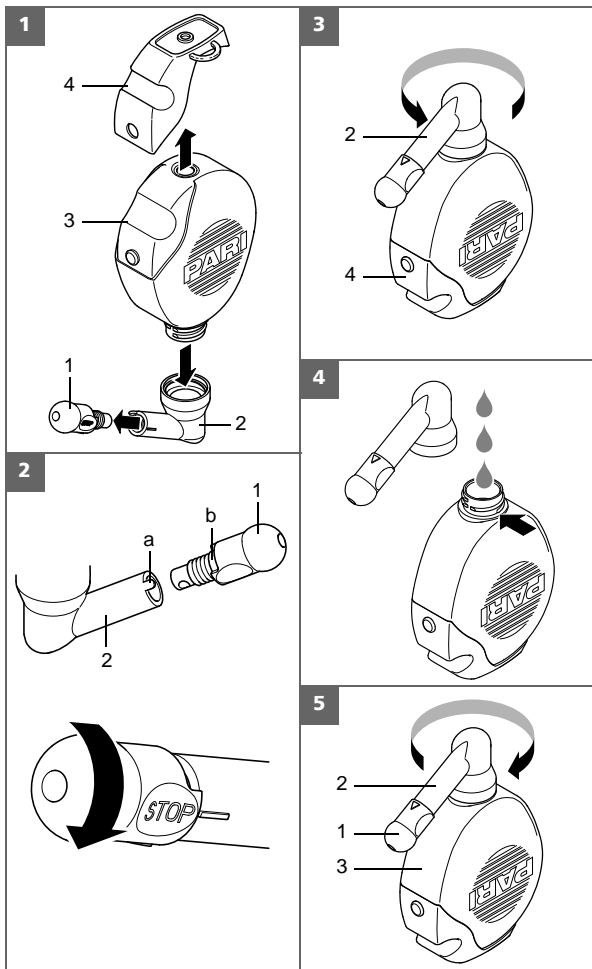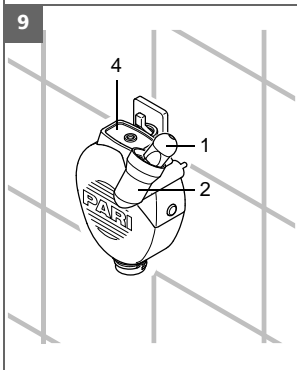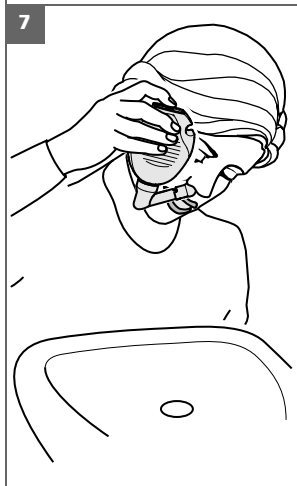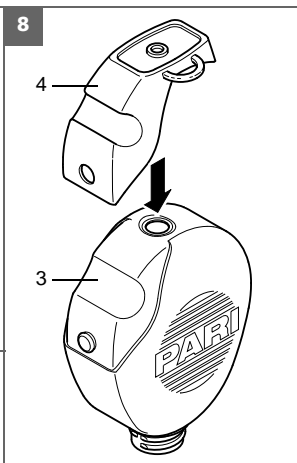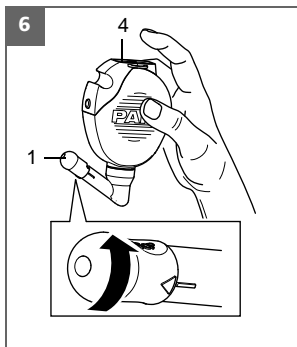en

Instructions for use

Read these instructions for use before using for the first time.
Keep these instructions in a safe place.

1 General Information

What is the PARI MONTESOL nasal douche and what is it used for?

The PARI MONTESOL nasal douche is a simple medical device that is used as an aid for flushing the main nasal cavity. The nasal douche moistens the mucous membrane in the nose and cleanses the nasal cavity.

The nasal cavity is flushed as a measure against:

- frequently occurring colds
- susceptibility to cold infections
- dry nose and nasal calculus
- very dusty and contaminated atmospheres
- pollen allergies
- chronic inflammation of the nose and nasal cavities
- and following surgical procedures in the area of the nose and nasal cavities (only if the doctor agrees)

The nasal douche is suitable for use by children aged 3 and older as well as by adults.

What materials is the PARI MONTESOL nasal douche made from?

Polypropylene	Bend, reservoir
Thermoplastic elastomer	Nasal insert, valve

What must I do before I use the PARI MONTESOL nasal douche for the first time?

- Boil the PARI MONTESOL nasal douche thoroughly before using for the first time and prior to disposal (see chapter "Hygienic re-use", page 14).
- Replace the PARI MONTESOL nasal douche with a new one every 12 to 18 months.
- **When treating babies, children and anyone who requires assistance, the following must be observed without fail:**
The product contains small parts. Small parts can block the respiratory tract and pose a choking hazard. Therefore, make sure that you always keep these parts out of the reach of babies and infants.

When must I not use the PARI MONTESOL nasal douche?

The PARI MONTESOL nasal douche must not be used:

- when you have open wounds in your nose. In this case, flushing would cause heavier bleeding.
- when you are unable to close your soft palate, e.g., because of palatine paralysis.

What do I fill the PARI MONTESOL nasal douche with?

Nasal rinses should always be carried out with isotonic (i.e. the concentration of the mineral content should match its concentration in the bloodstream) saline solutions.

It is very important to follow this instruction.

Concentrations that are too high or too low may cause unpleasant irritation.

The fill volume in the PARI MONTESOL nasal douche is 250 ml.

For nasal rinsing, please use suitable brine or saline solutions (e.g., PARI MONTESOL nasal rinse) which are available in liquid or crystalline form from your chemist or specialist medical dealer. Your chemist or specialist dealer will be glad to tell you the quantity you need to use to prepare 250ml of an isotonic solution. This quantity is added to the douche and warm water is added until the PARI MONTESOL nasal douche is completely filled.

Please also follow the instructions for use for the brine and saline solutions you use for nasal rinsing.

The water you need to prepare the saline solutions must be at least drinking water quality and should always be fresh. It may be necessary to use boiled or sterile water to treat particular illnesses such as cystic fibrosis or after surgery.

2 Preparation

Figure 1

Familiarise yourself with the nasal douche before using it for the first time.

The PARI MONTESOL nasal douche consists of 4 parts:

- (1) Nasal insert
- (2) Bend
- (3) Reservoir
- (4) Valve

How do I fill the PARI MONTESOL nasal douche?

Figure 2

- Place the nasal insert (1) on the bend (2) (already pre-fitted at shipment). Ensure that the small pin (a) on the bend has engaged in the slot (b) in the nasal insert.
- Turn the nasal insert to the STOP position (flow will be blocked): The STOP label on the nasal insert must be aligned with the marking on the bend (line).

Figure 3

- Set the nasal douche down with the valve (4) at the bottom.
- Unscrew the bend (2) from the nasal douche.

Figure 4

- Fill the nasal douche with the correct dose of brine or saline solution in accordance with the instructions for use.
- Fill the nasal douche up to the lower edge of the screw thread (see arrow) with drinking water at body temperature.

Figure 5

- Then screw the bend (2) firmly back onto the reservoir (3). Ensure that the nasal insert (1) is in the STOP position (see Figure 2).
- Turn the nasal douche over.
- Rock the nasal douche gently back and forth to mix the brine or saline solution thoroughly with the drinking water.

3 Application

Note:

You can rinse via either nostril. To rinse the left nostril, hold the nasal douche in your left hand. To rinse the right nostril, hold the nasal douche in your right hand. In the following, nasal rinsing via the right nostril will be described.

Figure 6

- Hold the filled nasal douche in your right hand; place your right index finger on the valve (4).
- With your left hand, turn the nasal insert (1) to the rinse position (flow symbol).

Figure 7

- Bend your head slightly forwards over the basin and open your mouth wide so that your soft palate closes.
- Breathe gently in and out through your mouth.
- Place the nasal insert for the PARI MONTESOL nasal douche over your right nostril, so that the nostril is completely sealed.
- Now turn your head slightly to the left.
- Press the valve (4) with your index finger, which is resting on the douche. Rinsing will begin. The nasal rinse solution flows in through the nostril, through the nasal cavity, and back out through your other nostril.
- You can stop rinsing at any time by taking your index finger off the valve. At the end of rinsing, a small quantity of the solution will be left in the nasal douche.

What should I do if I get rinsing fluid in my eyes?

Rinse your eyes with abundant fresh water.

What should I do if the rinse solution runs into my throat?

Small quantities of nasal rinse solution can always flow into the throat. If you tilt your head slightly forwards and to one side while rinsing, this small quantity will flow quite naturally out of your mouth.

If larger quantities pass into the throat, either your mouth is not open wide enough to close your soft palate, or the nasal passage on the other side is so congested that the rinse solution cannot flow back. In the first case, remember to open your mouth as wide as possible, and if necessary shift the position of your head slightly over the basin. If the problem is congestion, stop rinsing and begin again through the other nostril.

What should I do if the rinse solution continues to run out of my nose after rinsing is finished?

This happens with some users whose noses have many recesses. These users should shake their head in all directions over the wash basin after each rinsing session, so that all of the rinse solution runs out of the nose.

Do not attempt to keep unused residue of the rinse solution. Discard all unused solution. Otherwise bacterial contamination may occur!

4 Hygienic re-use

4.1 Cleaning

Inadequate cleaning or drying may allow germs to breed which could cause or aggravate illnesses.

The PARI MONTESOL nasal douche should therefore be cleaned, disinfected and dried after each use.

Recommended method

- Dismantle the nasal douche into its individual parts (see Figure 1).
- After every use, flush the individual parts of the PARI MONTESOL nasal douche thoroughly with warm drinking water, then disinfect and dry them (see next chapter).

Possible alternative: Cleaning in the dishwasher

- Distribute the parts in the cutlery basket.
- Select a programme of at least 50°C.

Do not wash the PARI MONTESOL nasal douche together with dirty crockery.

4.2 Disinfection

Recommended method

You can disinfect the PARI MONTESOL nasal douche in a steam disinfectant. Please follow the instructions for use with each of these steam disinfectants.

Possible alternative

The PARI MONTESOL nasal douche can be sterilised by boiling. Place the individual parts of your nasal douche in boiling water for 15 minutes. Do not leave the parts unsupervised. Ensure the parts are not touching the sides of the pan, and that there is sufficient water in the pan to cover them. After boiling, allow the parts to cool down and dry off.

4.3 Drying

Figure 8

- After disinfecting, shake the water out of all parts.
- Attach the valve (4) to the reservoir (2).

Figure 9

- Fit the bend (2) in the recess on the valve (4) provided for this purpose.
- Place the nasal insert (1) into the opening in the bend.
- Hang the nasal douche up to dry by the ring on valve.

5 Miscellaneous

5.1 Disposal

After it has been sterilised by boiling, the PARI MONTESOL nasal douche can be discarded with domestic waste.

5.2 Explanation of symbols

- Caution, see instructions for use
- Warning

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PARI GmbH