# VisuXL® Eye drops for premium relief and recovery of dry eye

Preservative free

## EN Instructions for use

### When should you use VisuXL®?

VisuXL® is an ophthalmic, lubricating and anti-oxidant solution without preservatives, composed of Coenzyme Q10, Vitamin E TPGS and cross-linked hyaluronic acid salt.

The cross-linked hyaluronic acid is a more stable and effective form of hyaluronic acid than the non-stabilized form and gives VisuXL® better viscoelastic, mucomimetic and moisturizing properties and is more effective over time.


The combination of cross-linked hyaluronic acid and antioxidant/anti free-radical substances provides a more effective treatment in cases of:

- dry eyes;
- alterations in the continuity of the corneal and conjunctival surfaces after refractive surgery, trauma from a foreign body and surgery;
- alterations in the ocular surface related to metabolic disorders (diabetes);
- environmental stress (smog, chlorinated water, altered micro-climates, smoke, air conditioning);
- intense and prolonged exposure to UV and ionizing radiation;
- prolonged use of computers, monitors, or the television.

### Mode of use

Instill 1-2 drops of VisuXL® in the conjunctival sac of each eye 2 times a day or according to the opinion of your ophthalmologist.


### How to apply VisuXL®

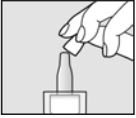Remove the cap.

Instill 1-2 drops into the conjunctival sac of each eye by pulling the lower eye lid down and looking upward. Dispense the product by gently squeezing the bottle by pushing down the dispensing tip.

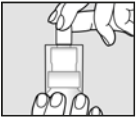Close the container immediately after use. The solution can be re-used within 6 months after the first opening.

### What other indications should be respected when applying the product?

In case of concomitant therapies, instill VisuXL® at least 5 minutes after the administration of other medications.

### Warnings

- Do not use this product if you are allergic or hypersensitive to any of the components;
- If you have any adverse reactions, discontinue its use and contact your ophthalmologist;
- The product can also be instilled while wearing contact lenses: it does not alter their characteristics, and can even hydrate soft contact lenses;
- Instill the product at least 5 minutes after the use of other drugs;
- The product maintains its sterility up to 180 days after the first opening of the bottle;
- The expiration date refers to the product in an intact, correctly stored package;
- Do not use this product after the expiration date indicated on the package;
- Wash your hands before applying the product;
- Do not let the tip of the bottle touch your eyes;
- Do not use the bottle if it is damaged;
- Keep the product away from light, heat and out of the reach of children.
- Store the product at room temperature (15-25 °C).
- Ophthalmic use, do not swallow.

### Composition

100 ml of VisuXL® contains:

- Coenzyme Q10 100 mg;
- Vitamin E TPGS (D- $\alpha$ -tocopheryl-polyethylene-glycol 1000 succinate) 500 mg;
- Cross-linked hyaluronic acid sodium salt 100 mg;
- Buffered isotonic solution up to 100 ml.

### Package

Bottle of 5 ml without preservatives in 3K System.  
Bottle of 10 ml without preservatives in 3K System.

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#### VISUFARMA B.V.
#### VISUFARMA S.p.A.,
#### Omisano Farmaceutici S.r.l.