Gebrauchsanleitung  
Manual  
Mode d'emploi  
Használati utasítás  
Manuale  
Gebruikershandleiding  
Instrukcja obsługi  
Manual de Instruções  
инструкция  
Návod

Merula® #  
menstrual cup

Die moderne Art der Monatshygiene ★ The modern kind of sanitary products ★ La forma moderna de productos sanitarios ★ L'hygiène menstruelle moderne ★ Modern menstruációs termék ★ Il moderno nei prodotti sanitari ★ De moderne manier van maandelijks hygiëne ★ Nowoczesny produkt higieny osobistej ★ O tipo mais moderno de produtos de higiene ★ Современное средство гигиены ★ Menstruační kalíšek to je moderní způsob menstruace

VEGAN

VEGANO

VÉGÉTALIEN

VEGÁN

VEGANA

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## IMPORTANT INFORMATION

Do not use the Merula Cup until you have completely read and understood the instruction manual. Store this instruction manual. This instruction manual is not a substitute for medical advice. If you have any doubts before, during or after use of the Merula Cup, please consult your doctor. The Merula Cup is a sanitary product and no returns and/or exchanges are allowed. All sales are final.

We reserve the right to make changes to the instruction manual without separate notice. You can always download the latest version of the Merula Cup instruction manual on our website, [www.merula-cup.de/en](http://www.merula-cup.de/en).

## Advantages of the Merula Cup

### Ecological

- ★ One cup can be used for several years
- ★ Produces far less trash than disposable products

### Economical

- ★ There is no need to frequently buy expensive supplies
- ★ After a few months, the small investment in a Merula Cup does pay

### Comfortable

- ★ Big capacity
- ★ Does not absorb water during swimming or in the bath tub
- ★ No thread dangling from the body (invisible)

## What are the particularities of the Merula Cup?

★ It has a very large capacity (up to 38 ml / 1.285 fl oz) with a diameter from 40 mm to 46 mm (1.575 in. to 1.811 in.). The Merula Cup adapts perfectly to your body and is an ideal companion for light to heavy bleeding.

★ From weak to strong pelvic floor muscles: The Merula Cup stays where it belongs. As it can be worn in a folded up state and does not have to open completely, even strong pelvic floor muscles do not make it lose its shape.

★ Due to its spherical shape, Merula Cup has a short height of only 39 mm / 1.535 in. and an impressive capacity, which is roughly equal to 2 large tampons. Because of that, Merula Cup can easily be used by women with a low cervix. The cup can also be used without any difficulty if you have a high cervix, as it has a long, ladder-shaped stem.

★ Measuring the height of your cervix before using the cup for the first time is not necessary. Simply decide, which length of the stem would be perfect for you. If necessary, you can adapt it by cutting one or two levels from the stem.

## Insertion & Removal

### Punchdown-Fold

Push the rim of the cup at one point in direction to the inner bottom and fold the whole cup. As you do so, the cup becomes smaller and it is easier to insert.

The cup can be moistened with a little bit of water. You can also use *Merula Lube* (or another water-based lubricant), which makes the cup extremely easy to insert. Insert the folded cup and make sure that the rim opens again when inside. The bottom of the cup can stay collapsed. If the cup leaks, try to correct its position. The cup does not have to be inserted as deep as a tampon.

For removal, press the cup slightly downwards and hold the stem. Move the cup downwards, with slow zigzag movements, until you can reach the bottom. Press the bottom in order to remove the light suction. Now, you can remove the cup completely and clean it.

Should insertion or removal not function properly right away, please practice fully relaxed for a few cycles and, if necessary, take some breaks. You will find further advice on our website: [www.merula-cup.de/en](http://www.merula-cup.de/en)

## Cleaning

Boil the cup before the first use and after every menstruation. Use water with a pinch of vinegar in a pot or in a microwave and boil the cup for at least 5 minutes. The cup can be used up to 8 hours at once, but can also be removed and cleaned more frequently.

After 8 hours, empty the contents of the cup into the toilet and rinse the cup with water. In order to prevent staining, rinse the cup with cold water first. Then, before reinserting the cup, clean it with some warm water and *Merula Spray* (designed to gently clean and disinfect your Merula Cup) or another disinfectant that is suitable for a sex toy. If you're out and about and have to empty your cup in a bathroom without a private sink, we recommend using our handy *Merula Wipes* for cleaning. As a last resort, you can also thoroughly rub and wash out your cup with some bottled water. But be sure to clean it with *Merula Spray*, *Merula Wipes*, or a comparable disinfectant as soon as possible afterwards, for it must be cleaned with one of these products at least once every 8 hours.

Keep the cup always in a container that is permeable to air, such as the bag that is delivered with Merula. Well maintained, a single cup can be used for several years. If the silicone begins to deteriorate, throw it away and buy a new one. Slight staining due to frequent use is normal.

## Adaptation

First, try your Merula cup. After that, you can adapt the stem to your individual length.

Remove the parts that are highlighted with nail scissors to adapt the stem to your required length. The cup should disappear completely inside your body, but still be easily reachable with at least one finger.

## Safety instructions

- ★ Always wash your hands thoroughly before and after using the cup.
- ★ Always loosen the light suction before removing the cup.
- ★ The Merula Cup should be removed before sexual intercourse and is neither suitable for contraception nor does it protect against sexually-transmitted diseases.
- ★ Clean the cup at least every 8 hours and boil it for a minimum of five minutes or more at least once after the end of the menstruation.
- ★ If you wear an intrauterine device (coil/chain/ball), consult your doctor beforehand on using a menstrual cup.
- ★ Dispose of the cup properly in the trash and not in the toilet.
- ★ Always store your cup out of the reach of children and animals.
- ★ If you have any doubts or experience pain while using the cup, remove it immediately and consult with a doctor.

### ★ Menstrual Toxic Shock Syndrome (TSS)

TSS is a rare but dangerous disease, which can occur with men, women, and children. At the moment, there have not been many known cases to be connected with a menstrual cup, but still: extreme cases of TSS can lead to death.

The following symptoms, which can develop very quickly and appear to only be influenza in the beginning:

Sudden high fever ( $>39^{\circ}\text{C}$  /  $>102^{\circ}\text{F}$ ), nausea, diarrhea, sunburn-like rash, sore throat, vertigo, and/or fainting.

If you experience one or more of these symptoms during your menstruation, remove the cup and consult with a doctor immediately. Mention TSS and that you are menstruating. This way, it is possible to diagnose the disease early and treat it effectively. If you are already affected by TSS, consult with your doctor before using cups or tampons.

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