| Artikel Nr. | Artikel | Größe/Form |
|-------------|--------------------------|------------------|
| MED1000986 | Rapssamenkissen gelb | 12 x 16 cm |
| MED1000987 | Rapssamenkissen gelb | 22 x 22 cm |
| MED1000988 | Rapssamenkissen gelb | 40 x 20 cm |
| MED1000989 | Rapssamenkissen gelb | 40 x 30 cm |
| MED1000990 | Rapssamenkissen gelb | 40 x 40 cm |
| MED1000991 | Rapssamenkissen gelb | 40 x 50 cm |
| MED1000992 | Rapssamenkissen gelb | 40 x 60 cm |
| MED1001091 | Rapssamenkissen gelb | 40 x 80 cm |
| MED1000993 | Rapssamenkissen blau | 'Fisch' |
| MED1000994 | Rapssamenkissen rot | 'Herz' |
| MED1001048 | Rapssamenkissen rot | 'Maus' |
| MED1000995 | Rapssamenkissen gelb | 'Mond' |
| MED1000997 | Rapssamenkissen rot | 'Nackenhörnchen' |
| MED1000996 | Rapssamenkissen gelb | 'Stern' |
| MED1000984 | Rapssamen Nachfüllbeutel | 1 Kg |

M E D E S  G N

medesign I.C. GmbH · Dietramszellerstr. 6 · D-83623 Dietramszell
 Phone +49(0)8027-907 38-0 · Fax +49(0)8027-907 38-27
 info@medesign.de · www.medesign.de

RAPSSAMENKISSEN

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RAPE SEED BAG

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RAPE SEED BAG

Bag for Gentle Massage and Thermo-Therapy

medesign's Rape Seed Bag is a traditional household remedy used in the same way as a hot water bottle. Therefore it also goes by the name of waterless hot water bottle. When the Rape Seed Bag is heated up, the pits inside the Bag will retain heat much longer than water in a hot water bottle. Due to its design, the Rape Seed Bag will perfectly adapt to the body's shape allowing for perfect thermal transmission.

Heating:

In the oven at a maximum temperature of 100° C. In the microwave, preferably without grilling or hot air function (danger of combustion!). Start with the lowest level and increase gradually until the cushion has been heated up to a pleasant temperature. Please check your microwave manual. Where less heat is needed, the cushion can also be laid on the radiator.

Cooling:

medesign's Rape Seed Bag may also be used to provide cooling. Simply put the Bag in a freezer or refrigerator.

Furthermore, the medesign Rape Seed Bag can be used in massages during pregnancy or for general hand and foot exercises. This is especially helpful when mobility of the extremities has to be restored through specifically applied massage, or activated in children.