## Mouth swabs en

The easy to use Leukoplast Oral Swab can help maintain oral hygiene. The hexagonally shaped swab head is made from an absorbent porous polyurethane presented as either a hard or soft foam. The design of the edges in the assists with the gentle removal of plaque, mucus and plaque. The swab can gently stimulate oral tissue and clean the teeth and gums. The swab is compatible with a wide range of oral cleaning liquids such as aqueous mouthwashes, saline and aqueous based antiseptics.

### Instructions for Use

This will guide on the proper usage and maintenance of the oral swab to help effectively maintain oral hygiene or provide care for the mouth and/or gums. Ensure hands are clean and dry before handling the oral mouth swab. Unpeel the pack of 5 oral mouth swabs and remove one from packaging, making sure to avoid touching the tip. Gently wet the swab's sponge tip with the clinically specified liquid, if desired. Do not oversaturate. Open mouth slightly and hold the swab like a toothbrush, ensuring a comfortable grip. Starting at one side of mouth, gently rub the sponge tip along gumline, making circular motions. Move the swab across the inner surfaces of cheeks, under tongue, and on the roof of mouth. Continue swabbing all areas of mouth, applying gentle pressure while maintaining comfort. Dispose of the used swab in a waste container or as per local regulations. The oral mouth swab is intended for individuals who do not have severe dental or gum diseases. If use results in any pain, bleeding, or discomfort consult a clinician. Not intended for the administration of medications to infants or small children.

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	Artwork ID: JBR15043.03	Review Loop: 2	Printing Colours:  Pantone 287 C
Project Title: C3081_LP clean-up Surgical Dressing mouth swabs		Date of Review Loop: 29.05.2024_RB	
Contact: ArtworkManagementMedical@essity.com		Replaces Artwork ID: JBR15043.02	
Contact Address: BSN medical GmbH Quickbornstrasse24 20253 Hamburg   Germany		Technical Drawing for print ID: -	
Languages: en, de, fr, es, it, nl, pt, se, no, dk, fi, id		Note LOT and expiry-date symbol will be part of variable coding.	

# ESKO WebCenter Approval Information

**Project:** JBR15043.03  
**Approval Comment:** All below mentioned approvers attest to the accuracy and integrity of this document by signing the document with digital signature implemented in WebCenter.  
**Project Name:** LP clean-up Surgical Dressing mouth swabs  
**Project Number:** C3081\_LP mouth swabs  
**Reason for Change:** new design - new diecut  
**Approval Email:** Dean Taylor:Dean.Taylor@essity.com, Kirstin Hammer:kirstin.hammer@essity.com, Kim Roettger:Kim.Roettger@essity.com, Sangar Srivaratharajan:Sangar.SRIVARATHARAJAN@essity.com  
**Document:** JBR15043.03.pdf  
**Document description:**  
**Version:** 2  
**Version ID:** 00002\_0000040234  
**Downloaded:** Jun 6, 2024 at 11:39 AM GMT+02:00  
**By:** Hilke Meyer (DEHHIME)  
**Approval state:**

## Approved

St age Nr	Approver	Role / Group	Date and Time	Action	Comment
1	Dean Taylor (DEHDETA), ESSITY - regular user (Germany)	PACKAGING ROLE	May 30, 2024 at 9:50 AM GMT+02:00	Approved (Final approval)	
1	Kirstin Hammer (DEHКИHA), ESSITY - regular user (Germany)	MARKETING ROLE	May 30, 2024 at 5:21 PM GMT+02:00	Approved (Final approval)	
1	Kim Roettger (DEHКИRO), ESSITY - regular user (Germany)	REGULATOR Y AFFAIRS ROLE	Jun 3, 2024 at 9:03 AM GMT+02:00	Approved (Final approval)	
1	Sangar Srivaratharajan (DEHSASJ), ESSITY - regular user (Germany)	MEDICAL ROLE	Jun 5, 2024 at 11:47 AM GMT+02:00	Approved (Final approval)	

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Multigate Co., Ltd.
Essity Distribution B.V. · Piet Heinkade 55