# tima-oculav.

Augenspülung  
Eye Wash  
Lavage oculaire  
Lavaggio ocular  
Irrigación ocular  
Oogspoeling

Sterile Phosphatpufferlösung  
Sterile phosphate buffer solution  
Solution tampon de phosphate stérile  
Soluzione tampone fosfato sterile  
Solución estéril tamponada con fosfato  
Steriele fosfaatbufferoplossing

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## Instructions for use - EN

First aid for rinsing the eyes, the conjunctival sac and the skin  
**Application - 1. Twist off the cap - 2. Rinse the affected area carefully - 3. Keep the eye open with the fingers while rinsing, if necessary.** Continue rinsing until all harmful substances are washed away completely. If required use additional bottles. If eyes are affected, remove contact lenses before application. Product is intended for first aid only. Consult a healthcare professional as soon as possible.

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EN

**Indication and clinical benefit:**

First aid for rinsing the eyes, the conjunctival sac and the skin:

After chemical burns caused by acids or lyes

In case of irritation (e.g. pepper spray or tear gas)

In case of foreign bodies

**Performance characteristics:**

Rinsing of the affected eyes and skin for removing irritating substances.

Phosphate buffer can neutralize acid and lye due to its buffering capacity.

**Shelf-life and storage:**

Do not freeze. For single use only. Sterility is not ensured in case of re-used or damaged bottles. Do not use after expiry date.

**Contraindications and Precautions:**

Keep out of reach of children.

Do not use in the case of penetrating eye injuries (e.g. caused by a splitter).

**Side-effects:**

Not known. In case of any serious incident that has occurred in relation to this medical device, report to the manufacturer or your competent authority.

**Composition:**

Sodium dihydrogenphosphate dihydrate 4.4 mg/ml,

Disodium hydrogen phosphate dodecahydrate 40.4 mg/ml, purified water

**Notes:**

For adults and children. Children must be supervised by an adult.

Overdose is not possible as application is limited for first aid use.

Disposal according to local requirements.

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