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## Instructions for use

Please read this information carefully because it will help you to use HYLO CARE® properly.

# HYLO CARE®

Sodium hyaluronate 1 mg/ml and dexpanthenol

For lubrication and support of healing of a damaged surface of the eye e.g. due to dry eyes or surgical procedures.

Sterile, phosphate-free eye drops without preservatives

### What is HYLO CARE®?

HYLO CARE® is a sterile, phosphate- and preservative-free solution for ocular use.

### Composition:

1 ml HYLO CARE® contains 1 mg sodium hyaluronate, dexpanthenol, citric acid anhydrous, sodium citrate and water for injections.

### What is HYLO CARE® used for?

For lubrication and support of healing of a damaged surface of the eye e.g. due to dry eyes or surgical procedures.

### When should you use HYLO CARE®?

To regenerate and maintain the ocular surface - cornea and conjunctiva – it must be kept permanently moist. Normally our tears fulfil this role. However, in some cases there are not enough tears, or their composition is disturbed. In these cases, the eye reacts with redness, itching, foreign body and burning sensation, or acute tiredness.

These symptoms of an inadequate supply with physiologic tears may occur in dry or air-conditioned environments, as well as result of medical treatment or – purely - due to age.

The epithelial cells of cornea and conjunctiva are continuously restored, and this process works best in a humid environment.

Therefore, after surgical procedures it is vital that the ocular surface is sufficiently lubricated to allow proper wound healing.

In all these cases it is advisable to apply intensive humidity to the eye by using HYLO CARE® in order to renew the physiologic balance of lubrication and moisturisation of the eye.


HYLO CARE® contains the sodium salt of hyaluronic acid, a physiologic substance of the eye but also of other parts of the body. Sodium hyaluronate has unique properties which allow it to form an even and long-lasting lubricating film on the surface of the eye without being rinsed away quickly. Consequently, the moisturising effect is prolonged without impairing vision. Therefore, HYLO CARE® can be used during the day while eye ointments and gels can cause blurred vision because of their greasy consistency.

Additionally, HYLO CARE® contains dexpanthenol which belongs to the extensive group of B vitamins. This component assists the eye during its phase of intensive regeneration while the lubricating properties of sodium hyaluronate further support this process. In addition to this, dexpanthenol supports wound healing and so superficial erosions on the ocular surface caused by an insufficient tear film as well as wounds which are sometimes the result of surgical procedures, can both heal faster.

HYLO CARE® ensures not only a sufficient supply of humidity to the eye but at the same time enhances the regeneration of corneal and conjunctival epithelia.

### How should HYLO CARE® be applied?

Please do not use HYLO CARE® if any part of the protective seal on the folding carton is broken or damaged before using for the first time (☹).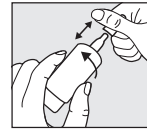


#### Step 1:

Remove cap before use.


To do this, hold the cap above the bottle. With your other hand, pull away the bottle from the cap with a rotary movement.

Before the first application please turn the HYLO CARE® bottle with its tip down and press the bottom of the bottle repeatedly until the first drop appears at the nozzle. Now the bottle is ready for further use.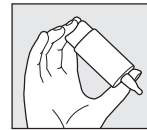


#### Step 2:

Hold the bottle upside down and place your thumb on the shoulder of the bottle and the other fingers on the base of the bottle.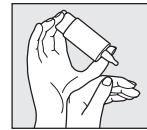


#### Step 3:

Support the hand holding the COMOD®-bottle with your other hand as shown.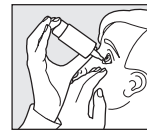

#### Step 4:

With your head leaning back slightly, gently pull down your lower eyelid. Press the base of the bottle vigorously and quickly. This activates the mechanism to release one drop. Because of the special COMOD® valve system, size and speed of each drop remain constant regardless of the pressure applied to the base of the bottle. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

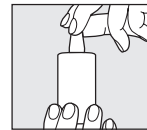#### Step 5:

Replace cap immediately and carefully after use.

Ensure that the dropper tip is dry.

During use, please avoid any contact between the tip of the bottle and your eye or your skin.

Each bottle releases 10 ml of solution which is equivalent to approximately 300 drops. For technical reasons a small amount of solution will remain in the bottle.

HYLO CARE® is only for you, even people with similar symptoms should not use your eye drops.

### What else do you need to know?

If you use any other eye drops there should be at least 30 minutes between the applications and HYLO CARE® should always be used last. Eye ointments should, however, always be administered after using HYLO CARE®.

Because it is preservative-free, HYLO CARE® is well tolerated even when used over a longer period of time. In very rare cases hypersensitive reactions like burning or excessive tearing have been reported which stopped immediately when the use of HYLO CARE® was discontinued.

HYLO CARE® may be used while wearing contact lenses. After inserting contact lenses, waiting for about 30 minutes before applying the eye drops is recommended. In doing so you avoid intolerance reactions which may occur due to interactions between lens care products and HYLO CARE®.

Based on the available evidence HYLO CARE® may be used in children, pregnant and breast-feeding women.

### How often and for how long can you use HYLO CARE®?

HYLO CARE® should be used according to your condition and the instructions given by your doctor or your pharmacist. Generally, you instil one drop of HYLO CARE® three times a day into each eye. If necessary, you may use it more frequently. If you use HYLO CARE® more frequently (e. g. more than 10 times per day) please consult your eye doctor. HYLO CARE® is suitable for long-term treatment. Nevertheless, as with all complaints you should consult your eye doctor if symptoms persist.

### When should you not use HYLO CARE®?

Do not use HYLO CARE® if you are hypersensitive to one of the ingredients.

### What you need to know about storing HYLO CARE®?

Do not store above 25 °C (↘<sup>25 °C</sup>).

HYLO CARE® can be used for 6 months after opening.

Do not use beyond the expiry date (☞).

Keep out of the reach of children.

The following packaging sizes are available:

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STERILE A

Medical device

Your opinion about our product is relevant for us!  
We are looking forward to get your feedback via mail  
or via [www.ursapharm.info/hylocare](http://www.ursapharm.info/hylocare).

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URSAPHARM Arzneimittel GmbH,