contrakel®

DE

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EN

Product type: Medical Device

Indications: Cream for treating hypertrophic and atrophic scars and cheloids, to be applied on intact skin, after the complete closure of the injury.

- Hypertrophic scars: they are frequent after-effects of burns, they are big, red-coloured and convex, they often itch.
- Atrophic scars: they are mainly after-effects of acne and German measles.
- Cheloids: they are after-effects of skin injuries, with irregular shape, they contract irregularly or even painfully.

Its composition, enriched with cholesterol and olive oil, softens the cicatricial tissue, thanks to the creation of an occlusive film helping to maintain the physiological hydration of the skin itself and to improve its aspect.

Besides, its composition contains:

- Allantoin: it stimulates the renewal of the skin and relieves the itch typical of arising scars.
- Urea: it helps the skin hydration.
- Vitamin A: it helps the correct functionality of skin cells and prevents an excessive keratinization.

Contrakel® Cream is suitable both for treating recent scars, generated after injuries, surgical operations, burns, chilblains, acne and for treating old scars.

Usage: Apply Contrakel® Cream on the scar by gently massaging for a few minutes. Repeat the application twice per day. The product can be used also for long periods of time, according to the dimensions and conditions of the scar. It is suitable also to be applied on delicate skins.

Warnings: Do not use if the package is open or damaged. The cream is only for external use. Do not apply on the mucous membranes or on not intact skin. Keep out of the reach and sight of children. Do not use on skin treated with antibiotics or other products for topical use. Do not use in cases of established or suspected hypersensitivity for one or more than one components present in the product. In case of undesired reactions suspend the treatment and consult a doctor. Reclose carefully the tube after the use.

Storage: Store in a cool and dry place, away from heat sources. The expiry date refers to the product in its original and integral package, properly stored.

Content: Tube 30 ml

Pharmadab, d.o.o.

Cesta na Brdo 100, 1000 Ljubljana, Slovenia, EU

info@pharmadab.com, www.pharmadab.com

Medical Device

Date of the last revision: 11/01/2017