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LUMBOFIX® MP**Stabilization bandage for the lumbar region of the spine**

Material: Nylon (57%), polyester (12%), latex (31%), velcro, perforated neoprene, stabilizers of aluminium, foam, steel springs.

Indications:

- Low back pain in degenerative and destabilizing diseases of the lumbar region of the spine (spondylosis, spondylarthrosis, osteochondrosis, invertebral hernia,pondylolysis/spondylolisthesis)
- Post-operative stabilization of the lumbar region of the spine
- Stable fractures of lumbar vertebral bodies
- Muscular and ligamentous insufficiency

Application:

- Remove the protective elements from the Velcro strips on the elastic side straps and at the right-hand end of the bandage.
- Position the bandage on the back so that the label is uppermost and the strip with velcro loops lies at the right-hand end of the bandage.
- Grasp both ends of the bandage between the middle fingers and the palm of the hand, and adapt to the front of the body, in the centre line. The two dorsal stabilizers should now be located symmetrically adjacent to the lumbar region of the spine, on the dorsal musculature
- The two separate side straps must be loose and hanging freely.
- Tighten the bandage evenly by pulling to the front in an upwards movement and close it. To do this, first fold the left-hand end over the centre line to the right, and then fold the right-hand end in the opposite direction to the left, and close by securing the strap with the velcro hooks to the pile section of the bandage
- Grasp the ends of the side straps and secure at the front, to the pile section of the bandage, applying the tension desired

Precautionary measures:

Do not use the product for patients with an allergy to latex.

The disposal of the device is the responsibility of the user. Local legal requirements should be observed. Store in a dry place.

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TRIAMED GmbH & Co. KG
Omnimed AG