TaloStabil® Plus Sport

Aktiv-Sprunggelenkbandage

Active ankle joint support | Chevillière active | Tobillera Aktiv | Cavigliera attiva

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The **ankle joint support Talostabil® Plus Sport from BORT** supports damaged tissue structures (joints, ligaments, capsules and cartilage). Here, the compression strengthens the retaining function of the ligaments, thus relieving the joint. At the same time, this support helps to avoid incorrect movement. When on the move, the compressed SOFTflex 3D knitted fabric ensures greater elasticity and stability and is very comfortable to wear, whilst the two silicone pads cause a massage effect which reduces the swelling tendency and contributes to faster regression of existing swelling. The Stabilo® strap for the **BORT Talostabil® Plus Sport** serves both additional stabilisation for inwards and outwards rotation of the foot. **COOLMAX®** fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

Instructions for use:

For stabilisation when rotating the foot inwards, that is with a raised outer edge of the foot (viewed from the rear, the strap points outwards):

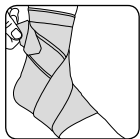
1. Guide the Stabilo® strap forwards diagonally over the metatarsal downwards to the inner side of the foot on the outer side of the ankle joint.

2. In the next step, guide the strap through underneath the sole of the foot to the outer side of the foot.

3. Then, draw the Stabilo® strap upwards diagonally over the metatarsal to the inner side of the ankle joint again. Raise the outer edge of the foot slightly here.

4. Then, fasten the Velcro tape onto the outer side from behind.

The tighter you tension the Stabilo® strap, the better the stabilising effect.

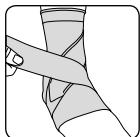
For stabilisation when rotating the foot outwards, that is with a raised inner edge of the foot (viewed from the rear, the strap points inwards):

1. Guide the Stabilo® strap forwards diagonally over the metatarsal downwards to the outer side of the foot on the inner side of the ankle joint.

2. In the next step, guide the strap through underneath the sole of the foot to the inner side of the foot.

3. Then, draw the Stabilo® strap upwards diagonally over the metatarsal to the outer side of the ankle joint again. Raise the inner edge of the foot slightly here.

4. Then, fasten the Velcro tape onto the inner side from behind.

The tighter you tension the Stabilo® strap, the better the stabilising effect.

Indications:

Chronic post-traumatic or post-operative soft tissue irritations in the ankle area, arthrosis, ligament weakness, after removal of plaster cast, ligament injuries, distortion of upper ankle joint, supination prophylaxis.

Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

Composition:

40 % polyamide (**COOLMAX®**), 40 % polyamide, 19 % elastane (**LYCRA®**), 1 % silicone

Important Note:

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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EN The **BORT TaloStabil® Plus Sport active support** relieves the weak or diseased ankle joint. Two viscoelastic pads stimulate circulation and help to reduce oedemas. The **SOFTflex 3D** knitted fabric and moisture regulating properties of the high-tech **COOLMAX®** fibre ensure that it is very comfortable to wear and for ideal properties for use during sports activities.

BORT Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | www.bort.com

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