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EN

Your physician has prescribed you with a high-quality therapeutic shoe, which is suited for early mobilisation in the form of this aid.

Please observe the following points in order to ensure that wearing this shoe is as comfortable as possible for you.

Instructions for use

The adaption of the BORT Diabetic Orthopaedic Shoe must be made by an expert (orthopaedic technician, orthopaedic shoe technician).

Protect the open injury on the foot with a bandage or similar. Use crutches when you wear the shoe. Always wear a non-slip shoe with the same heel height on the healthy foot.

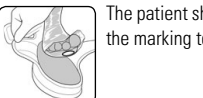
Honeycomb insole in the BORT Diabetic Orthopaedic Shoe

Insole for plantar pressure relief. The individually removable honeycombs allow individually adjustment – post-operatively or for acute wound care. A thin surface layer fixes the processed area with Velcro fastening and hinders the adjacent honeycombs from slipping. Pressure is reduced.

Application recommendation


Place a sterile film on the wound or the pressure ulcer (Image 1) and isolate the surface affected with a marker pen on the film (Image 2).

The patient should then stand on the inner sole in the shoe, thus transferring the marking to the uppermost blue layer of the inner sole (Image 3).

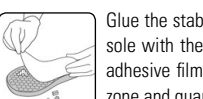Carefully remove the honeycombs beneath the marking where the pressure relief should be provided (Image 4).

Attention, do not remove more than 7 honeycombs per pressure relief zone, otherwise targeted pressure relief and pressure redistribution is not guaranteed!

Glue the stabilisation plate (included in the scope of delivery) fully onto the sole with the honeycombs. The stabilisation plate is equipped with a self-adhesive film, thus fixing and stabilising the honeycombs around the relief zone and guaranteeing that it remains stable and is not changed (Image 5).

Place the adapted honeycomb sole (with the stabilisation plate downwards) back into the shoe (Image 6).

Put it on and you're ready (Image 7).

Attention

We expressly recommend wearing bandage material or socks. If the product is not correctly handled, its function and optimum protection are no longer guaranteed. Extended hollows can result in fatigue symptoms and loss of function of the surface layer.

Indications

For targeted pressure redistribution in case of: plantar lesions and plantar ulcers, wounds in the foot and ankle region due to diabetes mellitus, peripheral arterial occlusive disease (PAOD), further wounds in the foot and ankle region, post-traumatic, post-operative.

Contraindications

Open fractures, extensive plantar wounds without adequate bandaging / covering of the skin in the wound area, always use with crutches.

You are not allowed to operate a vehicle when wearing the BORT surgical shoe!

Composition

Sole: 100 % EVA

Upper: 50 % PUR, 35 % Polyester, 15 % Polyamide

Knit loop / Velcro straps: 100 % Polyamide

Important Note

No reutilisation – this medical device is meant for the care of one patient only. Avoid hindering blood circulation, in case of numbness loosen Velcro closures and remove it if necessary. If complaints continue, please contact the doctor.

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