DryMax[®] Relevo[™] Instructions for Use

ENG

Intended use and indications:

DryMax/Relevo are sterile superabsorbent wound dressings for use on exuding wounds.

Product properties:

All these wound dressings have a superabsorbent core which absorbs and retains exudate. The exudate is absorbed vertically by the dressing from the white wound contact side and is retained in the core forming a gel, thus minimizing the risk of maceration to the peri-wound skin while maintaining a moist wound environment.

DryMax Foam/Foam Lite/Gentle/Sky have a hydrophobic coloured backing layer (side up) to minimize moisture strike-through. DryMax Border has a breathable water-resistant backing and a white wound contact layer with silicone adhesive for fixation. Apply smoothly to avoid creases on the border. DryMax Easy/Foam Lite have an adhesion function to make the dressing stick to the skin during the dressing application. Remove protection liner before application. DryMax Foam/Foam Lite have a polyurethane foam layer for a moister surface.

Instructions for use:

The dressings should be used on exuding wounds and handled by healthcare professionals. The dressings can be used as either primary or as secondary dressings. Clean the wound as per local protocol. Dry the surrounding skin carefully. Choose a size of dressing depending on the wound size; the core edge of the dressing should overlap the wound with a margin as the sealing edges do not absorb exudate. Apply the dressing with the white side in direct contact with the wound bed or the primary dressing. Secure the product in place. The dressings can be used in combination with compression therapy. The dressings should be changed according to the clinical condition of the wound, or before saturation. If the wound deteriorates, consult your healthcare professional. Dressing removal can be facilitated by moistening the dressing. Dispose of the dressing as clinical waste according to local regulations. In case a serious adverse event has occurred in relation to usage of any of the devices, this should be reported to Absorbent AB and the competent authority in the country in question.

Caution:

Do not cut the dressing, the cover must remain intact. Do not use the dressing if the sterile packaging is damaged. Single use product, do not re-sterilize, can not be re-used. Important to use the white wound contact side towards the wound! DryMax Foam/Foam Lite can be used on dry wounds for protection, but other DryMax/Relevo wound dressings should not be used on dry wounds.

Contraindications:

DryMax/Relevo wound dressings should not be used on eyes, mucous membranes, tendons or in wound cavities because the dressing swells during absorption. DryMax/Relevo wound dressings should not be used on persons with a known sensitivity to the dressing or its components.

Do not reuse. Reusing will cause cross contamination.

Do not use if the sterile barrier or packaging is compromised.

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