[DE] AIRMED'S NASENSAUGER**KINDERSCHNUPFEN**

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(V03, September 2024)

[EN] AIRMED'S VAC**CHILDREN'S COLD**

1. Colds are very common for babies
2. Babies cannot blow their noses themselves
3. Blocked nose prevents normal breathing, drinking, and sleeping
4. Consequential diseases of ears, sinuses or bronchi are possible
5. Removal of secretion and keeping the airways clear is therefore very important

EFFECT OF AIRMED'S VAC

1. Airmed's Vac was specially developed by a doctor for treatment at home
2. Nasal secretion is completely and gently removed in a few seconds
3. Baby can breathe, drink and sleep freely immediately
4. Effect was proven in a study by the University Children's Clinic Vienna
5. Airmed's Vac is suitable for children from birth
6. Special construction reduces the suction strength to baby-friendly nose blowing
7. Airmed's Vac is compatible with every vacuum cleaner (possibly with adapter)

ASSEMBLY & USAGE

1. Place suction head A on part B
2. Connect part C to the vacuum cleaner
3. Switch on the vacuum cleaner, regulate suction if necessary
4. Hold the child's head as still as possible
5. Guide suction head A to the nostril
6. Suck for a few seconds until secretion is removed
7. Repeat the process on the 2nd nostril

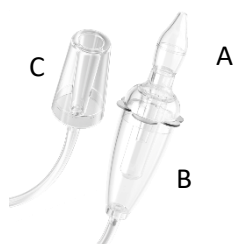
Use small parts only under adult supervision

CLEANING

1. Take parts A and B apart
2. Clean suction head (A) under running water with a brush
3. Sterilize suction head in the sterilizer (**Attention: No pressure sterilizer!**)
4. If dirty: Rinse part B with running water
5. If dirty: Rinse hose with running water
6. Do not leave the parts in washing liquid

LEGAL NOTES

1. Intended purpose: Cleansing the nasal passages in infants (from birth) and small children when their nose is blocked
2. Benefit: Facilitates free breathing and thus drinking and sleeping behaviour. Reduces the risk of secondary diseases (middle ear infection, sinusitis, bronchitis)
3. Contraindication: In case of frequent nosebleeds, contact your doctor in advance
4. Can be used with vacuum cleaners with a suction power of 800-2,000W
5. Store the device at room temperature in a dry place. Protect it from direct sunlight and heat
6. Study: Application study of a technically identical nasal aspirator in May 2005 by the University Clinic for Pediatrics and Adolescent Medicine at the Medical University of Vienna
7. For questions and reports in accordance with MDR 2017, please contact: Reshape Healthcare Würzburggasse 16 A-1130 Vienna Austria office@airmedsvac.net

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