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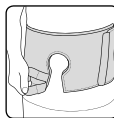
EN

The BORT Stoma Support provides support in case of abdominal tissue infirmity especially also in the area of artificial intestine exits. In the anus praeter area the pressure pad provides for stabilisation of the affected tissue structures.

Instructions for use:

If used with a pad, firstly attach it with Velcro on the inner side of the support in the region of the stoma recess.

Put the support on loosely. Loosen both Velcro fastenings for the stoma recess.

Position the opening around the stoma region.

Guide the stoma bag through the opening and close both Velcro fastenings again.

Finally, close the support so that a sufficient, pleasant stabilisation effect is achieved.

Double Velcro fastening for easy placement and positioning of the stoma recess, respectively on the left and right in the abdominal region.

Accessories available:

Art. no. 104 099 Stoma protector

Indications:

Paralysis and hernia of abdominal wall, post-operative, condition after ileostomy and colostomy with accompanying weakness of connective tissue.

Contraindications:

Lymphatic drainage disorders including unclear soft tissue swelling, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body, latex allergy.

Composition:

56 % polyamide, 25 % viscose, 13 % cotton, 4 % elastodien, 2 % elastane

Important Note:

No reutilisation – this medical device is meant for the care of one patient only.

Wear support only when under strain, not during rest or sleep. Avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

The product contains latex and can trigger allergic reactions.

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