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# Instructions for use


Congratulations on your purchase of the BORT AktiVen® Donning Aid for medical compression stockings! This aid enables you to put on your compression stockings in an energy-saving and comfortable manner, even in case of reduced strength and restricted mobility, thus helping you to handle your compression stockings safely and independently in daily use.

## **Indications**


Considerable functional disorders of the finger, hand or arm joints, primarily the gripping function, highly inflammatory or considerably degenerative joint diseases, paralysis, deforming diseases in the hand region, results of injury, amputations, malformations, far-reaching spinal, hip and knee joint stiffness as well as obesity.

## **Fitting instructions**


In order to ensure successful use of your donning aid from the beginning, we kindly request you to read these instructions for use attentively. Use the BORT AktiVen® Donning Aid for medical compression stockings together with a table and chair with four freestanding chair legs.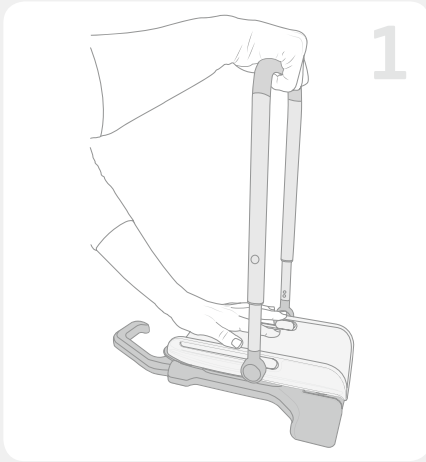


Put the BORT AktiVen® Donning Aid for medical compression stockings on the table. Whilst holding the shell with one hand, pull the frame to its greatest possible length with the other hand until the side locking mechanism engages.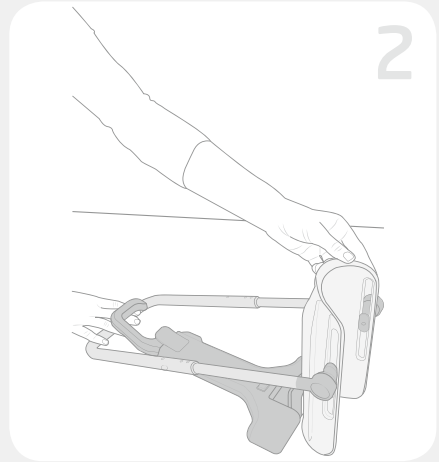

Place the shell in an upright, vertical position and the frame at the rear of the table.

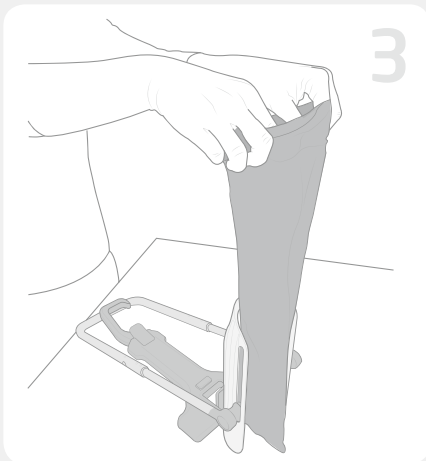
Grasp the compression stocking with both hands at the upper edge so that the heel points towards the body.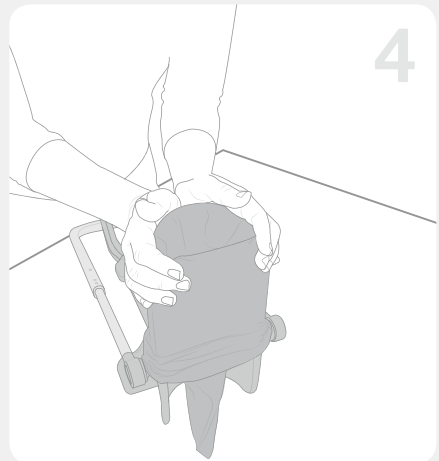

Then slip the compression stocking evenly over the shell until the stocking's heel area is around the upper edge of the shell. **Recommendation:** For easier and energy-saving placement of the stocking, use BORT AktiVen® Special Gloves.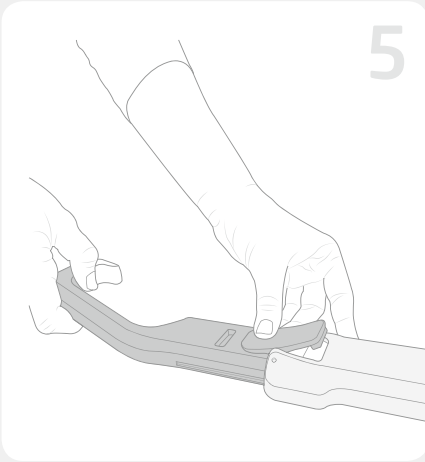

Set the splint hook so that the distance between the hook and the rear edge of the shell is approximately as long as your foot.


Put the shell down on the base plate and set the frame upright.


Sit down on the chair with sufficient space in the immediate vicinity and place the donning aid with the base plate on the floor. Anchor the hook around the leg of a chair, securing the donning aid against forwards movement.

Continue to hold the frame and insert the foot into the compression stocking from the rear to the heel.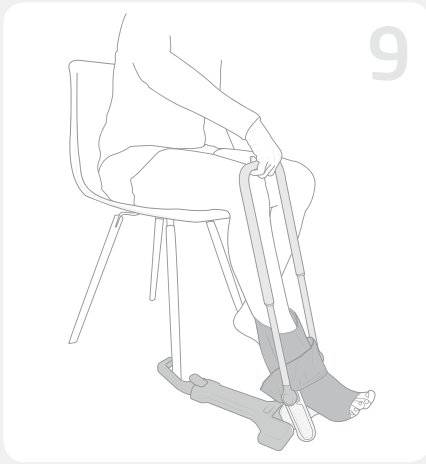

Press your foot further forwards against the resistance of the compression stocking.

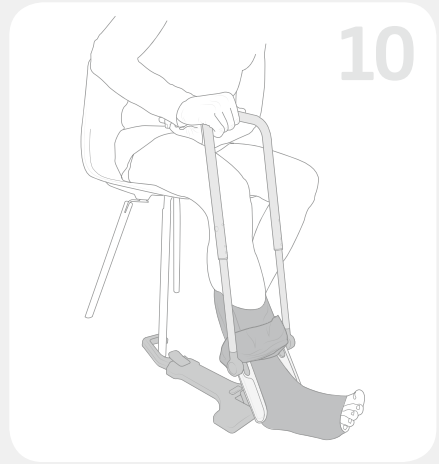
As soon as the shell straightens up, press your foot down until your heel contacts the floor.

Press your heel onto the floor and pull the frame in your direction until the upper edge of the shell reaches your popliteal fossa.

Place the frame aside.

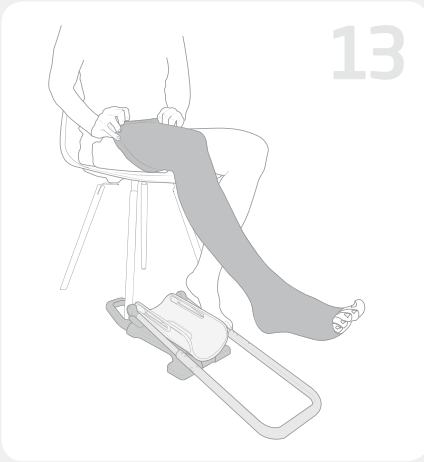Pull the compression stocking the rest of the way to the desired position.

If required, the knitted fabric can still be ideally spread over the leg and any creases in the compression stocking smoothed out. It's best to use the BORT AktiVen® Special Gloves for this step.

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**EN****Cleaning information**

To clean, wipe with a damp cloth. Do not use any chemical cleaning agents. Neither grease nor oil the articulated joints.

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## BORT Special Adhesive Lotion

- For the safe fixing of BORT AktiVen® medical compression stockings
- Skin-friendly, can be removed from the skin without any traces using water

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## BORT Special Detergent

- Specifically for knitted medical compression fabrics
- Top purity when hand-washing

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## BORT AktiVen® Special Gloves

- Simplifies putting on and taking off medical compression stockings
- Very good adhesion for perfect distribution and positioning of the knitted fabric

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**For daytime use – BORT WELEDA Venadoron**

- Cares and invigorates your legs
- Ideal before putting your compression stockings on

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**For nighttime use – BORT WELEDA Sea Buckthorn Body Lotion**

- Nourishes dry skin lastingly preserves its moisture content
- Ideal after wearing your compression stockings

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\*products available in Germany and Austria only

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BORT GmbH