# EpiBasic® Sport Ellenbogenbandage

Elbow support | Coudière | Codera | Gomitera

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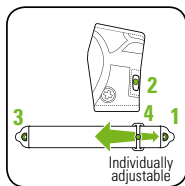
The **elbow support EpiBasic® Sport from BORT** supports damaged tissue structures (joints, ligaments, capsules and cartilage). Here, the compression from the support strengthens the retaining function of the ligaments, thus relieving the joint. The SOFTflex 3D knitted fabric ensures more elasticity and stability and is very comfortable to wear. The silicone insert provides a massage effect which accelerates the healing process. The targeted pressure has a pain-reducing effect and relieves strain on the tendon attachments. **COOLMAX®** fibre wicks moisture away from the body, ensuring dry and pleasant wearing comfort.

### Instructions for use:



Place the support on the elbow affected and position the knitted comfort zone in the antecubital fossa. The support is correctly positioned if you can feel the outer and inner condyle through the hollows in the silicone insert.

Stabilo® strap with the easyClick system for individual pad pressure dosage:

- 1 and 3 attachment module
- 2 supporting module
- 4 adjustment module

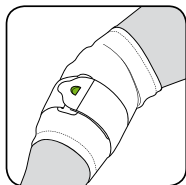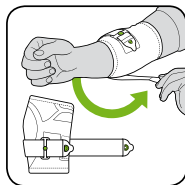Adapt the pad pressure dosage individually by moving the adjustment module (4).

When you receive your support, the attachment Stabilo® strap module (1) is attached to the support in the supporting module (2).

Guide the Stabilo® strap circularly around the lower arm under slight tension.

Close the strap by attaching the free attachment module (3) to the adjustment module (4). Pay attention to comfortable fitting of the Stabilo® strap, neither too loose nor too tight.

If the Stabilo® strap with the easyClick system is not needed, the supporting module (2) can be taken from the velour bag.

**Indications:**

Chronic post-traumatic or post-operative soft tissue irritation in the elbow joint area, distortion, epicondylitis (acute or chronic), tendomyopathy.

**Contraindications:**

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders including unclear soft tissue swelling distal to the positioned aid, sensory disturbances and circulatory disorders in the specific body region, skin diseases in the specific part of the body.

**Composition:**

40 % polyamide (**COOLMAX®**), 40 % polyamide, 19 % elastane (**LYCRA®**), 1 % silicone

**Important Note:**

No re-use – this medical aid is intended for treating one patient.

If the support is to be worn at night, observe that the circulation is not impeded. In case of feelings of numbness, loosen the support and remove it if necessary. In case of persistent complaints, you should consult a doctor.

Neither use any creams nor ointments when wearing the support as they could damage the material.

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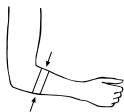
xx-small	20 – 22 cm
x-small	22 – 24 cm
small	24 – 26 cm
medium	26 – 28 cm
large	28 – 30 cm
x-large	30 – 32 cm
xx-large	32 – 34 cm

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Pull into shape and air-dry. Avoid heat.

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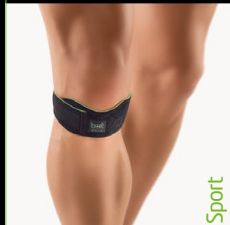
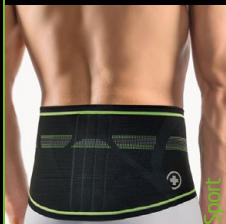
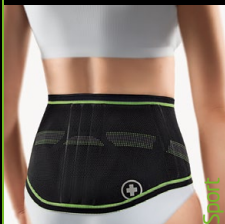
**EN** The **BORT EpiBasic Sport compression support** is used to support and relieve the afflicted elbow. Two viscoelastic pads stimulate circulation and relieve the painful tendon insertions. The pad pressure can be individually and easily metered using the Stabilo® strap with its easyClick system. The SOFTflex 3D knitted fabric and moisture regulating properties of the high-tech **COOLMAX®** fibre ensure that it is very comfortable to wear and for ideal properties for sports activities.

**BORT Quality products** promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.

BORT GmbH | Postfach 1330 | D-71367 Weinstadt | [www.bort.com](http://www.bort.com)

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