# DERMATOP SILVERSPRAY

EN | Hyaluronic acid sodium salt  
Metallic silver  
Vitamin E

## Indications

Treatment of uninfected skin lesions (e.g. abrasions, grazes, cracked skin, cuts, small surgical wounds, localized first and second degree burns) and superficial lesions in diabetic foot, athlete's foot and pressure sores. Suitable for adults and children (3+).

## How it acts

**DERMATOP SILVERSPRAY** is a suspension spray for topical use containing hyaluronic acid, metallic silver and vitamin E. The hyaluronic acid and silver are combined in a matrix (kaolin and starch) with absorbent properties. Absorption of the exudate creates an environment that facilitates the natural tissue regenerating process. In addition, the presence of hyaluronic acid allows the formation of a protective barrier against microbial penetration. Also contains Vitamin E with antioxidant properties.

## Dose and method of use

- 1) Wash the lesions before applying the spray.
- 2) Shake the container well before use for 15-20 seconds.
- 3) Holding the container in a vertical position, spray a quantity of powder sufficient to cover the surface to be treated at a distance of about 15 cm from the skin for 2-3 seconds. In the event of temporary jamming of the can, shake it vigorously until you hear the sound of the metal ball inside it.
- 4) After application, do not massage the affected area; the powder must form a continuous and protective layer.
- 5) We recommend the medicated part be covered with sterile gauze.
- 6) Apply one to three times a day or as advised by your doctor, depending on the clinical progress.
- 7) It is easily removed by washing the lesion with saline.

There is no actual danger associated with possible overdose of the product, as the use of the spray is only topical.

## Warnings

External use: do not swallow or inhale. Avoid contact with the eyes, if contact occurs, rinse with plenty of water. Avoid exposing the treated area to the sunlight. Avoid direct contact between the spray container and the wound. In case of undesirable reactions, stop treatment immediately, consult your doctor and inform the manufacturer. Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority of the Member State where the user and / or patient is established. There are no known interactions with medicines and medicinal products. Do not use in conjunction with other therapies or treatments in

place that involve taking or topical use of other medical devices or drugs. Consult a doctor for prolonged use. Close the package after use. Do not use if the package is damaged or after its expiry date. Consult your doctor before using the product in pregnant and breastfeeding women. Do not use the product for more than 30 consecutive days.

## Composition

Main ingredients: hyaluronic acid sodium salt (0.25%), kaolin + corn starch (93.7%).  
Other ingredients: metallic silver, vitamin E, rice oil, terpineol, disiloxane, LPG propellant.

## Contraindications

Do not use in case of known hypersensitivity or known allergy to one or more components.

## Storage

Store at a temperature between +5°C and +30°C.

Danger: Extremely flammable aerosol. Pressurized container: may burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not spray on an open flame or other ignition source. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of the reach of children.

The lot no. and expiry date are indicated on the bottom of the container.

The expiry date refers to the undamaged product correctly stored.

Contains no ozone-depleting propellant gases.

After use, dispose of in accordance with the local regulations in force.

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Fidia Farmaceutici S.p.A.