**mamivac®**  
■■■■ **EASY**

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CE

### Intended use:

For expressing breast milk in case of breastfeeding problems, illness or separation of mother and infant.

Observe safety instructions

Transportation and storage: -25°C to +70°C  
Operation: +5°C to +40°C

Observe operating instructions

Protect from sunlight

Conformity mark according to  
EU Council Directive 93/42/EEC

Manufacturer

KaWeCo GmbH  
Gerlinger Str. 36-38  
71254 Ditzingen  
Germany

Phone: +49 (0) 71 56 / 1 76 02 - 100  
Fax: +49 (0) 71 56 / 1 76 02 - 500  
info@kaweco.de

**mamivac.com**  
**Made in Germany**

Before using the **mamivac® EASY** manual breast pump, read the operating instructions carefully and keep them for future reference.

## Contents

1. Notes on the operating instructions	11
2. Assembling and components	12
3. Purpose	13
3.1 Indication	13
3.2 Contraindication	13
4. General notes	13
4.1 Functioning of the manual breast pump	14
5. Hygiene and cleaning	14
6. Practical tips	15
6.1 Storing breast milk	15
6.2 Accessories	15
7. Classification	16
8. Disposal	16
9. Spare parts	16
10. Warranty	16

## 1. Notes on the operating instructions

These operating instructions are valid for the **mamivac® EASY** manual breast pump as well as all other derivatives of identical design.

## 2. Assembling and components

a) Pump unit with vacuum control

b) Hygienic cover

c) Soft insert

d) Breast shield


e) Silicone valve

f) Bottle

g) O-ring seal

h) Bottle cover

i) Piston seal ring

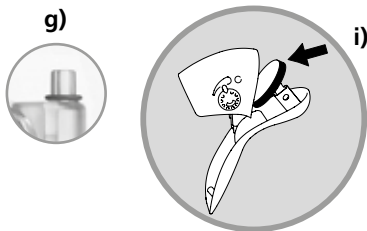### 3. Purpose

The manual breast pump **mamivac® EASY** is used in order to gently and softly express and collect breast milk.

#### 3.1 Indication

Breastfeeding problems that require the use of a breast pump may be: mastitis (inflammation of mammary glands), aching or sore mamillae (nipples), anatomic variants of the mamillae which make feeding the infant difficult (flat or inverted nipples), insufficient or excessive milk production, premature delivery, metabolic disorders of the infant, anatomic disorder in the face of the infant, poorly sucking infant, temporary illness of the mother (medication intake).

#### 3.2 Contraindication

The silicone soft insert is in direct contact with your skin. In case of a known silicone allergy, do not use the soft insert. In individual cases, the skin may be irritated resulting in redness or red spots. Taking medication in general may affect breastfeeding an infant. The patient information leaflets of the pharmaceutical companies contain appropriate information.

**mamivac® EASY may not be used:**

- to suction other body fluids, such as blood, secretions or mucus
- to suction outside the medical field, such as corrosive, flammable or explosive liquids
- on other living beings.


### 4. General notes

- Prior to expressing, wash your hands thoroughly with soap and wipe your breast with a damp cloth. Do not use soap or alcohol as detergent to avoid dry, soar or cracked nipples.
- Prepare yourself for the expressing with ease. Take a comfortable and relaxed seat. Gently press the breast shield over the nipple for a tight contact. Start to express milk.
- The pump has a stimulation mode to trigger the let-down reflex. You can also use your free hand for an additional breast massage.
- Express from both breasts alternately, unless instructed otherwise by the doctor. Since expressing times strongly depend on the individual mother, you are asked to contact your midwife or lactation consultant.

## 4.1 Functioning of the manual breast pump

### The stimulation phase (1):

To trigger the let-down reflex, turn the vacuum control to the very left. Press the pump handle dynamically downwards several times. This imitates the baby's natural sucking behaviour. Keep the pump handle pressed for 3-4 seconds, so that the milk starts to flow into the bottle.

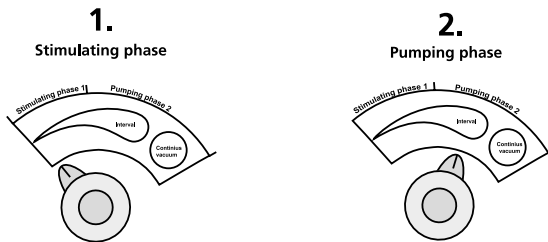### The expressing phase (2):

When the milk is flowing, turn the vacuum control to the right. You can express now by pressing the pump handle downwards and keeping the handle pressed until the vacuum has been released.

If the milk does not start to flow right away, relax and try again – even at a later point in time.

If expressing causes pain, do not use the pump anymore! Please contact your doctor or your midwife/lactation consultant. Make sure not to express too much milk into the bottle. Keep the pump in an upright position.

## 5. Hygiene and cleaning

Before you use the breast pump for the first time, boil components b) to h) in boiling water for at least 15 min. or use a steam disinfection device, such as a steriliser for instance. Please observe the manufacturer's corresponding notes.

**Do not use a microwave oven.**

- Do not use chemical disinfection baths, solvents or abrasives.
- All components b) to h) can be cleaned with commercial washing-up liquids by hand or in the dish washer.

- Prior to using the pump the next time, clean the parts which came into contact with milk, and check them for proper functioning and condition.
- After cleaning, rinse components b) to h) thoroughly with lukewarm water to remove residues of the washing-up liquid, especially on those components which come into direct contact with breast milk. You might use a brush to clean areas which are difficult to reach.
- Assemble the manual breast pump according to the drawing in »Assembling and components«.
- In case the piston is sticky, apply a drop of baby oil to the piston seal ring (i).

## 6. Practical tips



Wrap the cleaned and dry parts in a clean cloth and store them in a container when not in use.

### 6.1 Storing breast milk

**Breast milk can be stored as follows:**

- Freshly expressed at room temperature (16-29°C) 3-4 hours
- In the rear of the fridge (below 4°C) 3 days
- In the freezer (-18°C) 6 months

Fill the used bottles or containers to two thirds only and write down the fill date. Instead of a bottle, you can also use our specially designed **mamivac**® breast milk freezer bags [REF 282100]. Do not defrost frozen breast milk in the microwave, but at room temperature. Place the bottle or bag in warm water (37°C) as required. After defrosting, gently shake the container or bag to mix the milk.

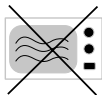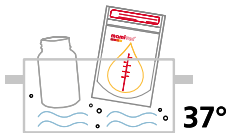### 6.2 Accessories

- Soft insert REF 281215
- **mamivac**® breast milk freezer bags PU = 20 pcs. REF 282100
- Hygienic cover REF 49010147

## 7. Classification

Classified according to Council Directive 93/42/EEC: Class I.

## 8. Disposal

Please dispose of your **mamivac® EASY** manual breast pump in accordance with your local disposal regulations.

## 9. Spare parts

Silicone valve	REF	B49054105
Piston seal ring	REF	49054125
O-ring seal	REF	49054005
Hygienic cover	REF	49010147
Standard breast shield	REF	28012011
Bottle with blind cover, screw ring and cover	REF	28011115
Retaining spring	REF	49040100

## 10. Warranty

Ignoring the notes in these operating instructions or an improper use of the device will void any warranty claims. KaWeCo GmbH rejects any liability for personal injuries and material damages arising from this.

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