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Operating Instruction for the Herpifix

Please read before using and keep carefully.

Dear User,

Thank you for deciding to purchase a Herpifix to treat your cold sores (Herpes labialis). This device is licensed as an individual medical product. For hygienic reasons it should only ever be used by one person and not passed on to others. We recommend that before you use the Herpifix you check with your physician that your problem really is Herpes labialis. These instructions have been made as detailed as possible to ensure your safety and to enable you to obtain the best possible results. Please follow them carefully. Should you have any questions please contact your physician or pharmacist. We hope that the Herpifix will help you to remove your problem quickly and safely.

The following general information on Herpes labialis will help you to understand the problem and how the Herpifix can help you to deal with it. The Herpes simplex virus is extremely common. Most people will become infected at some time. Once this has happened the virus stays in our body for the rest of our life. Normally we are not aware of this but certain conditions (UV-light, deficiency in our immune system, infections, etc.) may enable the virus to become more active with the result that we start to suffer from cold sores, mainly around the lips. These take the form of blisters which eventually form scabs.

At present there is no vaccine that will completely destroy the Herpes virus. Thus any treatment, including the Herpifix, can only treat the symptoms, not the underlying cause. Even where the treatment is successful, it is always possible

that the problem will reappear. Even the Herpifix cannot guarantee 100% success.

How the Herpifix works

The Herpifix is based on a completely new treatment concept. By using an electrical field with a very low and completely harmless voltage, we can change the acidity (pH) of the skin of the lips and the underlying tissue from the normal value (around 5.5) to below 2. Not only does this reduce itching but it also creates a virostatic effect. If the Herpifix is used correctly then the virus is prevented from spreading. Its ability to cause sores is either reduced or completely prevented, as is the creation of sores or scabs. According to the latest scientific research, reducing the skin's pH locally in this way is entirely harmless and reversible. The skin will generally return to its normal state without any adverse effect. The voltage was selected to ensure absolute safety for the user. However, as the change in skin pH will depend upon skin resistance it is possible that some people will not see the normal effect when using the Herpifix. The compact design of the Herpifix means that you can keep it with you at all times. We hope you will be satisfied with the Herpifix and its performance.

Application

The Herpifix is designed to treat Herpes labialis (cold sores) in the early development.

Safety Instructions

Herpifix should only be used on undamaged lip skin. You should not use the Herpifix where a disease or circumstances imply that a medical device should not be used without first consulting a physician. To provide you with proper advice it is important that you tell the physician about former diseases, other treatments you are

receiving at present and any special conditions or habits that you may have. You should not use the Herpifix where sores are already present or scabs have formed. You should not use the Herpifix when wearing lipstick. If you are fitted with a cardiac pacemaker you should consult your physician before using the Herpifix.

For hygienic reasons it must only be used by one person and not passed on to others.

For individually marking the Herpifix, please use the enclosed colour rings.

Keep the device out of the reach of children.

Side effects

Side effects with the Herpifix should be minimal. If the Herpifix is used correctly then these will be limited to slight pain and a feeling of tightness in the lips. This should disappear quickly.

Correct use

Most people suffering from Herpes labialis feel a tightness in the lip region before the disease appears as blisters. This is the correct time to use the Herpifix. Take the device out of the box

and remove the protective cap. Check that the ring electrode on the end of the device is neither bent nor damaged in any way.

Note: Normally the current must be adapted to your individual skin resistance. To avoid a resetting of the current by mistake, the switch was made hard to turn on purpose.

Please use a suitable coin at the place marked in the image to put the switch into the desired position. Do not use your fingernail to move the switch.

Set the voltage to the level you need. We suggest you start with level 2 (central position). If your lips are very sensitive start with level 1. Now press the button and release quickly. If the Herpifix is working correctly the green control diode will light up. Wet your lips with your tongue. Now press the Herpifix lightly on to the lips at the spot where you feel the tightness. We recommend that you use a mirror if at all possible. Ensure that both the central and ring electrodes make good contact with the skin.

Hold down the ON button while watching the green diode. It should be lit, but not as bright as before. You may feel a slight tingling of the lip. This is normal due to the change of pH in the skin. Keep the Herpifix pressed lightly on the lips for 3 seconds, even if you do feel the tingling, then remove it. The Herpifix will turn itself off automatically after 30 seconds.

If you feel acute pain when using the Herpifix then remove it immediately. Set it to a lower level and try again after 10 – 30 seconds. If you still feel pain please read the chapter on »Problems«. If you feel nothing when using the Herpifix then set it to a higher level, wait 10-30 seconds, then try again. If you have the Herpifix set at the highest level and still feel nothing please consult the chapter on »Problems«. Do not perform more than 3 treatments per day on the same site. After treatment you may notice a slight swelling of the lips. This is normal and should disappear within one to four hours. After using the Herpifix replace the protective cap.

Warning: The Herpifix is designed only for the treatment of Herpes labialis. We cannot accept any responsibility for its use for other purposes.

Image 1

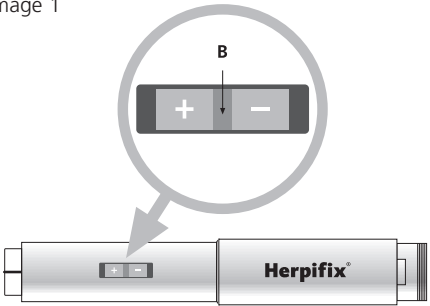
Image 2

Image 3


Description

- A** ring electrode with middle pen
- B** control button
- C** diode
- D** button (On/Off)
- E** battery cover

Setting up and changing battery

Take the Herpifix and the supplied battery from the package and insert it carefully, + pole first (image 4). Replace the battery cover. Close it by a right rotation of the cover using an appropriate coin. Remove the protective cap and press the button. The diode should light up. The Herpifix is now ready to use. Should the diode not light up please refer to the section on »Problems«. To replace battery open the battery cover by rotating left with an appropriate coin (image 4). Never put on the battery cover without battery as it will be difficult to open it. Use only batteries similar to that supplied. Please dispose of used batteries in the correct (environmentally friendly) way.

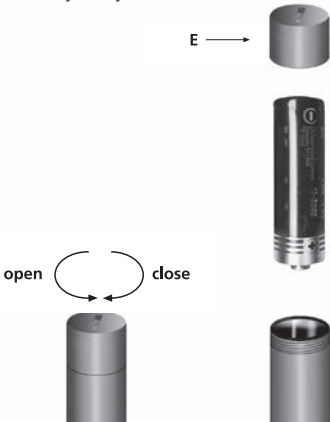Image 4

Cleaning

To clean the electrodes use a cotton wool bud and alcohol. Be careful not to bend the electrodes. Should you not be using the Herpifix for a long period we recommend that you remove the battery and store it separately. Protect the Herpifix from water, heat and strong sunlight.

Problem	Cause
Diode does not light when button is pressed	Battery is exhausted
	Battery is not inserted correctly
	Electronic system is defective
Electrodes are bent	
Even when using device on the lowest setting you feel pain	You may be pressing the device too hard on the lips
	Lips are damaged
	Device is defective
	You have a very low skin resistance
You feel no tightening of the lips when using	The device is defective
	Insufficient contact between electrodes and lip
	Your skin resistance is too high
Technical Data	
Battery	1,5 V micro / AAA
Treatment voltage	0,15/0,20/0,25 mAmp (+/- 10%)
Maximum voltage	20 V
Treatment levels (max.)	Level 1 : 3,0 mWatt, Level 2 : 4,0 mWatt, Level 3 : 5,0 mWatt
Max. power	max. 90 mWatt
Operation test	Green control diode

Action

Change battery

Open cover and check.
If necessary reinsert battery

Do not use

Dispose of device

Do not press so hard, but ensure that the electrodes to make contact with the skin

Check and if necessary consult your physician

Return for checking

Do not use

Have device tested

Wet your lips again and use slightly more pressure

Do not use

Permissible temperature extremes

10–40 °C

Device life

Approx. 1,400 applications with one battery

Product should be brought to the return and/or separate collection systems available to end-users, when the product has reached the end of its lifetime. This symbol applies only to the countries within the EU (EEA).

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Conditions of Guarantee

The perfect condition and performance of this Courage + Khazaka electronic appliance are guaranteed for a period of 2 years. The guarantee period starts with the date of sale to the purchaser.

We guarantee the elimination of all production and material failure. The guarantee will be fulfilled either by repair or replacement of faulty parts at our option.

The guarantee does not apply to damage to parts subject to wear, to damage and effects caused by improper treatment or incorrect maintenance, nor for the supplied battery.

The conditions of guarantee only apply if either the Guarantee Card with date of purchase, dealer's stamp and signature, or the receipt of purchase is sent to Courage + Khazaka electronic together with the device.

Transportation costs have to be paid by the customer.

This guarantee note is in addition to the consumers statutory rights and does not affect those rights in any way.

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TÜV Rheinland Product Safety GmbH, Cologne, Germany