# BIOPSY PUNCH

(EN) Read instructions carefully before use.  
 (AT) Packungsbeilage vor der Anwendung sorgfältig lesen.  
 (DE) Lesen.  
 (ES) Leer cuidadosamente las instrucciones antes de utilizar.

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(EN)  
 → For use by medically trained professionals.  
 → Do not re-sterilise.  
 → Do not re-use due to risk of infection.  
 → Do not use if the sterile blister package is compromised or damaged.  
 → Device must be used under aseptic conditions  
 → Wear gloves while handling device and packaging  
 → If multiple biopsies or samples are needed on the same patient, a new device must be used each time

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- 3.
- (EN) The Biopsy punch must never be reused and must be properly discarded in a puncture-resistant sharps bin as biologically contaminated material, in accordance with local protocols

**REPORTING OF SERIOUS INCIDENTS**  
Report any serious incident in relation to the Biopsy punch to the manufacturer. You should also report any serious incident in relation to the Biopsy punch directly via the national reporting system to the appropriate authority (see details below). It is important to report such incidents as this can help identify faults with this medical device and may prevent similar incidents happening again. United Kingdom: MHRA Yellow Card Scheme at: [www.mhra.gov.uk/yellowcard](http://www.mhra.gov.uk/yellowcard)  
Ireland: HPRA Pharmacovigilance, Website [www.hpra.ie](http://www.hpra.ie)

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- (AU) GlaxoSmithKline Australia Pty Ltd, Level 4, 436 Johnston Street Abbotsford, VIC 3067, AUSTRALIA.
- (AT) GlaxoSmithKline Pharma GmbH, Wien, AUSTRIA.
- (CH) GlaxoSmithKline AG, Talstrasse 3-5, CH-3053 Münchenbuchsee, SWITZERLAND.
- (DE) GlaxoSmithKline GmbH & Co. KG, 80700 München, GERMANY.
- (ES) GlaxoSmithKline, S.A., P.T.M. C/Severo Ochoa nº2, 28760 Tres Cantos, Madrid, ESPAÑA.
- (FI) GlaxoSmithKline Oy, Piispanilta 9A, PL 24, 02230 Espoo, SUOMI.