## SkinNeutrAll®

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## SkinNeutrAll®

## Notes and instructions for use

Please read carefully before use and follow the instructions!

Immediately after arrival of the spray, insert the bottles into the spray head!

**SkinNeutrAll®** is delivered with separate bottles and individual spray head. Please insert bottle 1 and bottle 2 in the spray head immediately after arrival. To do this, leave the two bottles in the outer carton and open the bottles first (unscrew the screw cap). Then insert the spray head with the two suction hoses:

- Insert bottle 1 with gray marking on white suction tube and  
- Insert bottle 2 with red marking on white suction tube with red reducer.

The two bayonet locks on the spray head are placed crosswise and then the spray head is placed on the bottles until they noticeably click into place. Then the caps are turned a quarter turn so that the bottles are fixed in place.

Now place **SkinNeutrAll®** within easy reach in the hazard area. It can now be used immediately at any time.

1

## SkinNeutrAll®

### Instructions for use

Only use both bottles together in the intended spray device!

Spray affected areas several times until all visible areas are covered. Allow to act for at least two minutes and then wash or wipe off as far as possible.

A red coloration of the sprayed area shows that oxidizing agents have been inactivated.

**Avoid eye contact!** In case of eye contact, rinse immediately with water or eye rinsing solution and seek medical advice. If possible show the label of the bottles!

**Do not spray into open wounds or onto damaged skin!**  
**Do not spray on mucous membranes!**

Do not use for longer than the stated expiration date (at least two years after delivery) / max. storage temperature 35°C.

**Attention!** If SkinNeutrAll® is still to be used after application or accidental actuation of the spray lever, clean the spray head. To do this, remove the spray head, place it in a glass of clear water and press the spray lever several times. Then continue spraying without water until all parts of the spray mechanism are free of liquid again. If the spray head is not cleaned, the mixture of the two liquids in the spray head can lead to crystallization and failure of the mechanism.

2

## SkinNeutrAll®

### What is SkinNeutrAll®?

SkinNeutrAll® is an emergency spray for use on uninjured skin, clothing, and objects by instructed persons.

It is a medical device certified in the European Union for neutralizing uninjured skin and clothing after contact with oxidizing agents, acids, bases, aldehydes and for precipitating fluorides for short-term use.

### How does SkinNeutrAll® work?

SkinNeutrAll® contains ascorbic acid, an amino acid buffer and a calcium salt as active agents. Glycerin and polyethylene glycol are solubilizers.

SkinNeutrAll® reduces strong oxidizing agents (such as chlorine, hypochlorite, chlorine dioxide, chloramine T, bromine, iodine, fluorine, hydrogen peroxide, peracetic acid) to compounds that are no longer active, such as chloride, and neutralizes acids and bases. Open explosives are inactivated. Formaldehyde is bound as imine. Furthermore, fluorides are precipitated to sparingly soluble calcium fluoride. Organic solvents and chemicals are dissolved and heavy metal salts are bound as complexes.

Much more information can be found at  
[www.skinneutrall.com](http://www.skinneutrall.com)

3

## SkinNeutrAll®

### Are there any restrictions on use?

SkinNeutrAll® can be used without hesitation even if the chemical in question is not known. SkinNeutrAll® actively inactivates or neutralizes acids, alkalis, oxidizers, fluorides, aldehydes and phenols. Organic solvents and chemicals as well as heavy metals are dissolved and removed from the uppermost layers of the skin and textiles by the solubilizers and the hypertonic preparation. There are no restrictions for the application.

### Are there any side effects?

SkinNeutrAll® has been dermatologically tested for its skin compatibility. It shows no undesirable skin reactions (allergies) and no skin-irritating effect if the instructions for use are observed (application on uninjured skin, exposure time not longer than one hour). If undesirable effects are observed, the manufacturer's safety representative (address at the end of the leaflet) should be contacted immediately.

### Is SkinNeutrAll® a hazardous substance or dangerous goods?

SkinNeutrAll® is not flammable, not corrosive or harmful to health and therefore not a hazardous substance or dangerous good. The spray can therefore be transported and kept ready in all buildings, vehicles aircraft and ships.

4

## SkinNeutrAll®

### How is SkinNeutrAll® disposed of?

Please dispose of unused solutions and incompletely emptied bottles according to official regulations. Waste code: 200130 (Municipal waste, cleaning agents). Do not dispose of the contents of the bottles in wastewater!

### Sources of supply:

Single sprays of SkinNeutrAll® are available from specialist retailers (EAN number 4280001912106) and pharmacies (PZN 15405458). Quotations for larger quantities should be requested from the manufacturer ilma biochem GmbH (info@ilma-biochem.com).

### Responsible manufacturer:

SkinNeutrAll® is subject to the European surveillance system for medical devices and is officially registered under the number DE/CA33ILM/2020/1/1. Further advice as well as reports of adverse effects to:  
Dr. Martin Weber  
Medical Devices Advisor and Safety Officer  
ilma biochem GmbH  
Loreleyblick 4-6  
56348 Dörscheid/ Germany  
Tel.: +49 175 9144607  
martin@ilma-biochem.com

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