DEUTSCH

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ENGLISH

INSTRUCTIONS FOR USE

FROXIMUN® TOXAPREVENT® SKIN Suspension is a Class I non-invasive medical product used to bind histamine locally on the skin.

1. Indications and usage

Binds histamine and exudates on the surface of the skin, particularly in cases of:

- skin irritation
- non-specific itching as well as reddening of the skin
- insect bites/stings (**Not a substitute for an anti-allergen!**)
- joint or muscle pain
- non-specific skin problems with intact dermis

2. Dosage and duration of use

Assistance from another person may be necessary for proper application of the suspension, especially if your range of motion is restricted or use the suspension in places that are difficult to reach. Unless otherwise instructed by a physician:

- Apply the suspension in a 2 mm film to the affected area of your skin and cover with an appropriately sized piece of medical foil.
- Use only medical foil to cover the suspension. Other foils may contain plasticizers that can cause skin irritations and adversely affect your health.
- The maximum application period is two hours.
- Remove the suspension after use by cleaning the treated site with clean, clear water.
- The suspension can be applied up to two times daily. Repeat the application only after you have cleaned the treated site.
- Unless otherwise instructed, after 14 days of repeated application you should discontinue application for 5 days.

3. Please note

- **This is not a substitute for an anti-allergen!**
- The suspension may be used only for the purpose specified in these instructions for use.
- Do not apply the suspension to open wounds.
- Stop using the suspension if you notice any unusual effects. If necessary, consult a doctor or pharmacist.
- Do not let the suspension get into your eyes or into other bodily orifices, e.g. your mouth, nose or ears. Exercise particular caution when using the suspension near bodily orifices.
- When using the suspension on your face, prepare the foil accordingly by cutting out holes for your eyes, nose and mouth.
- After using medicated creams, salves, etc., wait at least 60 minutes before applying the suspension. In cases of doubt, consult your doctor or pharmacist about using this medical product together with other medications.
- Avoid contact with the used suspension and the wound.
- Do not open the tube with the suspension until you are ready to use it. Close the tube after removing the desired amount.
- Always use a clean medical spatula to apply the suspension (included with the 120 ml packet). Always clean and disinfect the spatula before and after use. Follow the instructions for use of the disinfectant.
- After removal from the tube, the suspension must be used only once.
- The foil and the suspension can be disposed of in the household waste after use.

4. Storage and expiration date

- Keep out of reach of children!
- Store in a dry place at a temperature below 30°C.
- Keep out of direct sunlight.
- Expiration date: See end of tube

5. Ingredients

MANC® (clinoptilolite), colloidal water

6. Medical product manufacturer

FROXIMUN AG, Neue Str. 2a, 38838 Schlanstedt, Germany
Phone: +49 39401 632-0, Fax: +49 39401 632-199
eMail: anwender@froximun.de

These instructions for use were last revised on: 2014-07-04

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