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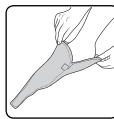
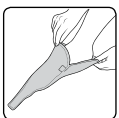
**EN**

The BORT Helix S Spiraldynamik® *USG*-Brace supports the natural torsional movement of the foot. The straps attached to the support act by exerting corrective traction on the heel bone towards the middle of the body and support the straightening of the foot arch. Due to the special guiding strap and the compression exerted by the brace, the item supports the holding function of the ligaments and relieves the joints while straightening the arch structures in their correct physiological position. At the same time, the brace helps to avoid incorrect movement. During movement, the knitted compression fabric and the silicone pads exert a massaging effect. The tendency for swelling is reduced and the healing process is accelerated.

#### Instruction for use:

Slip into the dressing aid with your foot.

Release the support strap. To ease putting the support on, pull it over the dressing aid on the foot. Check the correct fit.

Then, take hold of the dressing aid at the tip and pull it forwards out of the support. To simplify putting the rest on, we recommend – if possible – placing the foot on the thigh on the opposite side.

Guide the elastic strap around the ankle joint and over the instep to the outer surface of the foot.

Guide the strap underneath the sole of the foot to the inner surface of the foot. If necessary, the heel bone can be pulled towards the middle of the body using the other hand in this phase to support the restraints.

Close the elastic strap by placing the Velcro tip on the velour field intended for this purpose. Finally check and, if necessary, make corrections to the fit and the individual strap tension of the aid.

#### Indications:

Chronic, post-traumatic or post-operative irritations of soft tissues in the area of lower ankle joint and the sole of the foot, for example in the case of arthrosis, ligament weakness, disorders of the foot resulting from static defects and deformities of the foot caused by dysfunction of the lower region of the ankle joint, e.g. valgus foot, flat foot and metatarsus adductus.

#### Contraindications:

Peripheral arterial occlusive disease (PAOD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body, latex allergy.

#### Composition:

65 % polyamide, 28 % elastodien, 7 % viscose

#### Important Note:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor. While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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