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Ladysoft® Lubricant Gel

About one third of all woman suffer from insufficient formation of moisture at the vaginal entrance which may be one cause for pain during sexual intercourse. Experienced gynecologists therefore recommend Ladysoft® as moisturizing gel in order to compensate for insufficient lubrication. This is a natural way for you to experience unimpaired sexual intercourse. Ladysoft® also makes fits the wide range of application.

Special Properties of Ladysoft®

Ladysoft® may be used without hesitation over long period of time. It will not interfere with you hormone balance and is especially well tolerated by the skin. It has no contraceptive effects. Other fat-containing products, such as skin creams, lotions, etc. , should by no means be used as lubricants. Contrary to Ladysoft® these products may stick to the mucous membranes and cause irritations (of the skin). In addition, these products may also seriously damage condom material. Ladysoft® is absolutely neutral, fat-free, and leaves no stains.

Application

Before intercourse apply a thin layer of Ladysoft® to the vaginal entrance with your finger. When using a condom, slight wetting of its tip is sufficient. One to two drops will suffice in most cases. When using pessaries, Femcones, or Fembowls in order to strengthen the pelvic floor muscles, Ladysoft® facilitates introduction of the cones or pessaries. This will substantially improve training results.

Note

Ladysoft® should not be used if patients complain of pain or redness in the vestibule of the vagina or in the labia area.

In these cases a gynecologist should be consulted.

Side-effects have so far not been known.

Use by

Item Number (MED...)

Lot Number

Attention

Date of manufacture

Note Instructions for use

Manufacturer

CE marked with identification number the notified unit

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